


IN TALACE

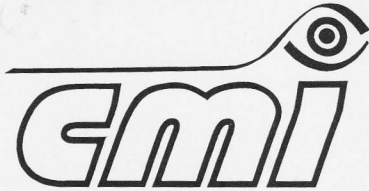

CMI spol. s ro.
 Seydlerova 2451, 155 00 Praha 5
 Česká Republika
 tel.: +420(2) 652 08 11, 652 08 41
 fax: +420(2) 651 99 01
 E - mail: cmisro@mbox.vol.cz

Servisní výkaz č.: 250701/1

Název podniku, adresa :

Fakultní nemocnice Olomouc
 oční klinika
 I. P. Pavlova 6
 Olomouc

PSC: 775 20

Datum: 25.7.2001

Číslo objed.:

Záruka do: 24.7.2002

Přístroj: Synoptofor Clement Clarke 2001

Popis závady a práce:

Instalace: vybalení, kontrola új. částí; nastavení
 nutných poloh, přepracování návodů
 funkce, instalace obložení
 Předem přepracován návod
 a nastavení v síti

... partner v kvalitě

Servisní technik:

Zóna:

Stupeň opravy:

Materiál:

Č	Název:	Katalóg.číslo	Model	Počet	Cena v Kč	
					za 1 ks	celkem
1	Synoptofor	SN20010262	2001	1		
2	44 ks sel. robitku	—	100/E	1		
3	Sada 12 obložení	6411001	2001	1		

Podklady pro fakturaci:

Spotřeba materiálu celkem:

Materiál:

Potvrzujeme převzetí dodávky do užívání.

Razítko a podpis:

Oprava:

Doc. MUDr. Jiří Řehák, CSc.
 přednosta oční kliniky FN a LF UP
 I. P. Pavlova 6
 775 20 OLOMOUC

Celkem:

SYNOPTOPHORE CHECKLIST

ISSUE 3 DATE : 07/06/99

Page 2 of 2

PREPARED BY: P.HALLYBONE

SCHEDULE REF	TEST	PASS/ VALUE
M	CYCLOPHORIA MINIMUM OF +/- 20 Degs	PASS
M	ELEVATION/DEPRESSION TORQUE TEST	PASS
N	BACKLASH IN CONTROLS	PASS
N	MOVEMENT IN CHASSIS	PASS
P	MOVEMENT OF CHINREST	PASS
Q	MOVEMENT OF HEADREST	PASS
Q	OPERATION OF ELECTRICAL CONTROLS	PASS
FUNCTION TEST		
D	PROJECTED VERTICAL LINES POSITION AND	PASS
E	PROJECTED HORIZONTAL POSITION AND ROTATIONAL	PASS
F	AFTER IMAGE LAMP TEST	PASS
H	AUTO FLASHING UNIT CHECKS	PASS
I	SLIDE ILLUMINATION CONTROL TEST	PASS
J	H.B. MOTOR CHECK (2001 ONLY)	PASS
K	SCREW TIGHTNESS CHECK	PASS
L	FINISH AND PRE-WRAPPING CHECK	PASS
ELECTRICAL SAFETY CHECK		
B	EARTH CONTINUITY Max 0.2 ohm	0.17 Ω
B	INSULATION Min 50 meg ohm	∞
B	EARTH LEAKAGE Max 0.5 mA	0.42 mA

CHECK LIST COMPLETED BY:  

SYNOPTOPHORE CHECKLIST

ISSUE 3 DATE : 07/06/99

Page 1 of 2

PREPARED BY : P.HALLYBONE

	MODEL : 2001
W/O No : 51317	SERIAL No : SN 2001 0262

SCHEDULE REF	TEST	PASS / VALUE
A	ARM LOCKS FUNCTIONING	PASS
A	OPTICAL TUBE HEIGHT DIFFERENCE (MAX 1mm)	= 0.54mm
A	OPTICAL TUBE FORWARD POSITION (MAX 1.5mm)	PASS
A	P.D. @ 45 (+/- 1mm)	= 44mm
A	P.D @ 75 (+/- 1mm)	= 74mm
B	FORCE REQUIRED TO MOVE ARMS (0.25 - 0.35 kg)	L = 0.28kg R = 0.27kg
C	P.D. TORQUE TEST (4oz MIN)	PASS
D	OPAL CHECK	PASS
E	SLIDE EJECTOR OPERATION CHECK	PASS
F	IRIS CHECK (2001 ONLY)	PASS
G	A.B.D. & A.D.D. MOVEMENT	PA
G	A.B.D. & A.D.D. TORQUE TEST	PASS
H	A.B.D. MOVEMENT FROM 0 TO 20 (10 Degs +/- 1)	PASS
I	25 Deg MINIMUM MOVEMENT	PASS
J	A.D.D. MOVEMENT FROM 0 TO 40 (WITH P.D. @ 47)	PASS
J	25 Deg MINIMUM MOVEMENT	PASS
K	A.D.D. MOVEMENT FROM 0 TO 20 (WITH P.D. @ 75)	PASS
K	25 Deg MINIMUM MOVEMENT	PASS
L	HYPOPHORIA MINIMUM OF +/- 10 Degs	PASS
L	HYPOPHORIA 1 oz TEST	PASS
L	HYPOPHORIA 3oz TEST	PASS