

Protokol o provedení periodické prohlídky

Servisní prohlídka dle doporučení výrobce, odpovídající požadavkům
zákona §27 zákona č. 346/2003 Sb. v platném znění

Zákazník Fakultní nemocnice Olomouc
Kardiochirurgická klinika

Přístroj EKG

Typ MAC 5000

Výrobní číslo 91MP 7071

Vyhovuje	Nevyhovuje	Provedené testy, zkoušky, měření.
☒	☐	Vizuální kontrola (mechanické části, kabely)
☒	☐	Auto test po zapnutí
☒	☐	Test ovládacích tlačítek
☒	☐	Test EKG obvodů, zobrazení amplitudy a periody QRS
☒	☐	Test tiskárny
		Bezpečnostní test – měření

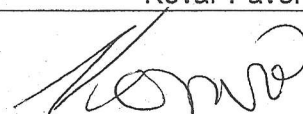
Měřená veličina		Naměřeno	Limit
Celkový unikající proud	Přístupná část	0,06 mA	max. 0,5 mA
	Příložná část	0,004 mA	max. 0,05 mA
Izolační odpor		≥ 20 MΩ	min. 20 MΩ
Odpor ochranného vodiče		0,08 Ω	max 0,2 Ω

Výsledek prohlídky (závěr) :

Parametry přístroje odpovídají – ~~neodpovídají~~ hodnotám předepsanými výrobcem. Zkontrolovaný přístroj je – ~~není~~ plně funkční a bezpečný zdravotnický prostředek pro další používání při poskytování zdravotní péče.

Použité měřicí přístroje:

Meditest 50	v.č.	200 341
-------------	------	---------

Technik	Kovář Pavel
Podpis technika	
Datum kontroly	20.6.2006
Další doporučená periodická bezpečnostně technická kontrola dle § 27 zákona č. 346/2003 Sb.	6/2007

Miscellaneous Setup

Institution name Ödalen, Kard och rurgis, Öllomouc
Text entry Uppercase only
Speaker volume Low
External video port Off
Information line Yes
Cart number 1
Site number 1
Location number 1
File Manager sort By Name
Delete after transmit Yes
Text on bottom No
Print barcodes Yes
Automatic Shutdown 10 minutes
Serial power always on No
System password *****
Overread password *****

System Configuration

MAC5000 005A (MM8413F)
ECG counter: 9630
Options
12SL*

Patient Questions

ID Required Yes Medications No
ID length 10 Referred by name No
Age No Referred by number No
Age in years No Test indication No
Gender No Patient history No
Height Yes Technician No
Weight Yes Location No
Height/weight in cm/kg No Room number No
Race No Options No
Blood pressure No Order number No
Secondary ID No
Extra questions
Prompt No
Type Numbers and letters
Prompt No
Type Numbers and letters

Service Only Setup

Serial number MMMP7071
Print head resistance 680
Keyboard English
Power Up Options
Power Up Application Resting ECG
Order Manager Enable
Initial sort value Patient name
Create orders locally Yes