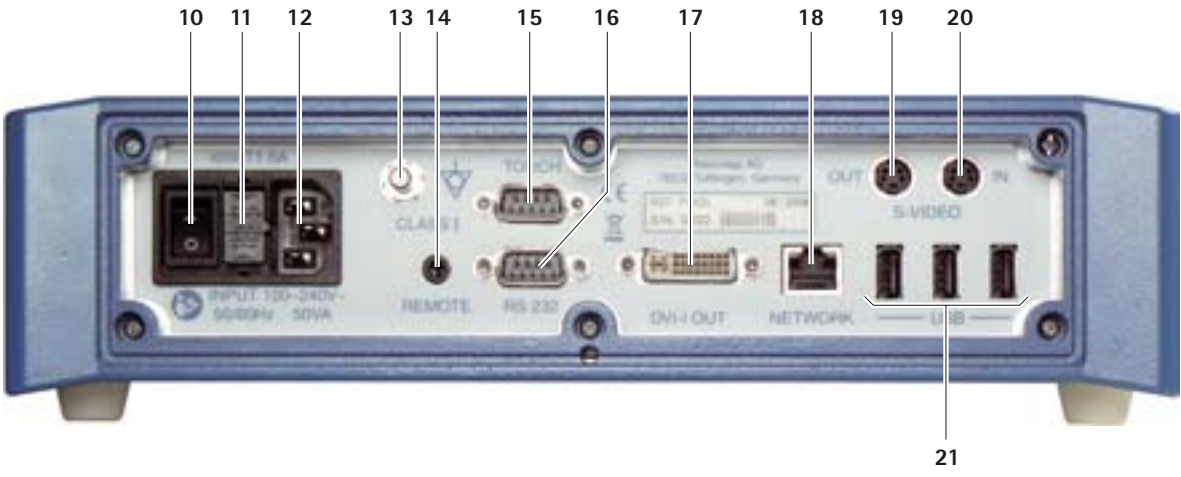
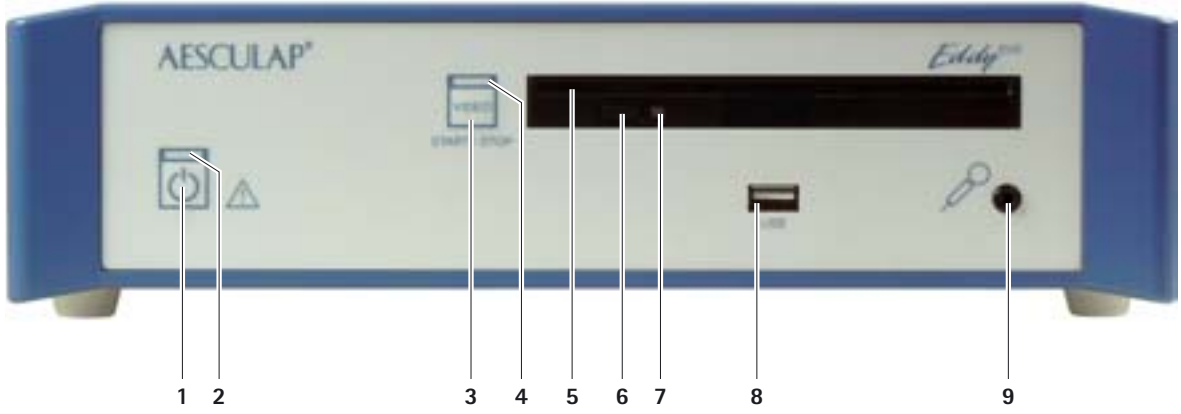
Aesculap Endoscopic Technology

Eddy DVD Digitales Dokumentationssystem

-
- (GB)** **Instructions for use/Technical description**
(USA) Eddy DVD digital documentation system PV820
 - (D)** **Gebrauchsanweisung/Technische Beschreibung**
Eddy DVD Digitales Dokumentationssystem PV820
 - (F)** **Mode d'emploi/Description technique**
Système de documentation numérique Eddy DVD PV820
 - (E)** **Instrucciones de manejo/Descripción técnica**
Sistema de documentación digital Eddy DVD PV820
 - (I)** **Istruzioni per l'uso/Descrizione tecnica**
Sistema di documentazione digitale Eddy DVD PV820
 - (CZ)** **Návod k použití/Technický popis**
Eddy DVD-digitální dokumentační systém

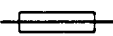
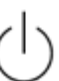
Eddy DVD digital documentation system PV820

Legend

- 1 On/Standby switch
- 2 On/Standby LED
- 3 Video recording START/STOP button
- 4 Video recording START/STOP LED
- 5 DVD/CD drive
- 6 EJECT button for ejecting the DVD/CD
- 7 Read/Write LED on DVD/CD drive
- 8 USB socket for external USB devices (e.g. hard disk, keyboard, optical mouse)
- 9 Microphone socket (3.5-mm jack)
- 10 ON/OFF switch
- 11 Fuse holder
- 12 IEC 320 mains power connector
- 13 Equipotentialization connector
- 14 REMOTE connector for remote control on the camera head
- 15 TOUCH connector (RS232) for touch-screen monitor
- 16 RS232 connector for DICOM dongle PV942
- 17 Video output DVI-I OUT
- 18 NETWORK connector (RJ45, Ethernet) for network cable
- 19 Video output S-VIDEO OUT (live image only)
- 20 Video input S-VIDEO IN
- 21 USB socket for external USB devices (e.g. hard disk, keyboard, optical mouse)

Symbols on product and packages

Symbol	Explanation
	CAUTION: General warning symbol CAUTION: See documentation supplied with the product
	Follow the instructions for use
	Equipotentialization connector
	Fuse
	On/Standby switch
	Microphone socket

Symbol	Explanation
	Labeling of electric and electronic devices according to directive 2002/96/EC (WEEE), see Chapter Disposal
	Date of manufacture

Contents

- 1. Safe handling 3
- 2. Product description 3
 - 2.1 System components 3
 - 2.2 Components necessary for use 3
 - 2.3 Intended use 3
 - 2.4 Operating principle 4
- 3. Preparation and setup 4
 - 3.1 First use 4
 - Positioning the Eddy DVD 4
 - Connecting the camera control unit 5
 - Connecting the monitor or touch-screen monitor to operate the software 5
 - Connecting a second monitor for live image display (optional) 5
 - Connecting the mouse 5
 - Connecting an external keyboard (optional) 5
 - Connecting a microphone (optional) 5
 - Connecting a printer (optional) 5
 - Connecting the mains power cord and equipotentialization cable 5
 - Stacking of units 5
 - Connecting the accessories 6
 - Connecting the power supply 6
 - 3.2 Software configuration in menu "Settings" 6
 - Menu "Settings/General" 7
 - Menu "Settings/Video" 8
 - Menu "Settings/Camera" 8
 - Menu "Settings/Printer/Network/System" 9
 - Menu "Settings/System" 13
- 4. Working with Eddy DVD PV820 15
 - 4.1 System set-up 15
 - 4.2 Function checks 15
 - 4.3 Safe operation 15
 - Explanation of the control elements on the screen 16
 - Inserting a DVD/CD 16
 - Creating a new patient 17
 - Creating a new surgeon or new operation 18
 - Finding patient data 18
 - Recording still images, video sequences and sound 20
 - Managing image data and changing patient data 22
 - 4.4 Switching off the Eddy DVD 28
- 5. Validated processing procedure 29
 - 5.1 General notes 29

5.2	Manual cleaning/disinfecting	29
	Wipe disinfection for electrical devices not to be sterilized.....	30
5.3	Inspection, maintenance and checks	30
6.	Maintenance	30
7.	Troubleshooting list	31
7.1	Messages on Eddy DVD	31
7.2	Other malfunctions	33
7.3	Changing fuses	35
8.	Technical Service	35
9.	Accessories and replacement parts	36
10.	Technical specifications	36
10.1	Ambient conditions	37
11.	Disposal	37
12.	Distributor in the US/Contact in Canada for product information and complaints	37

- To prevent damage caused by improper setup or operation, and in order not to compromise warranty and manufacturer liability:
 - Use the product only according to these instructions for use.
 - Observe all safety information and maintenance advisories.
- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge or experience.
- Keep the instructions for use accessible for the user.
- Always adhere to applicable standards.

1. Safe handling

CAUTION

Federal law restricts this device to sale by or on order of a physician!

Note

For operating the monitor and the camera system to be connected to Eddy DVD, refer to the instructions for use of the respective products. The present instructions for use only cover the handling and operation of the product "Eddy DVD PV820".

DANGER

Risk of death by electric shock!

- Do not open the product.
- Only ever connect the product to power mains with equipment grounding conductor.

WARNING

Risk of injury caused by incorrect operation of the product!

- Attend appropriate product training before using the product.
- For information regarding such training, please contact your national B. Braun/Aesculap agency.

- Thoroughly clean the new product after removing its transport packaging and prior to first use.
- Prior to use, check for proper condition and functioning of the product.
- Observe "Notes on Electromagnetic Compatibility (EMC)", see TA022130.

2. Product description

2.1 System components

Designation	Art. no.
Eddy DVD	PV820
DVI cable, 3 m	PV820201
Optical mouse	PV920200
CD containing Instructions for use and MPEG2 codec	PV920251
Instructions for use of Eddy DVD PV820	TA013091
Instructions for use of image viewer software PV943 for Eddy DVD	TA011486
Notes on Electromagnetic Compatibility	TA022130

2.2 Components necessary for use

In addition to the documentation system, the following components are required for documenting and visualizing a medical intervention:

- Country-specific power cord
- Monitor with DVI or VGA video input
- Video camera system with S-Video output

2.3 Intended use

Eddy DVD is a digital documentation system for displaying, recording, documenting, archiving and sending medical still images and video sequences on stationary or mobile data media.

Note

Eddy DVD may only be used for documentation purposes. The documented images are not licensed for use in diagnosis and/or therapy.

Eddy DVD digital documentation system PV820

2.4 Operating principle

The video signal is read in through the video input (parallel to real-time display through the video output) and saved as BMP- or JPG-format still images or as MPEG2-format video sequences.

The data are saved locally on the hard disk or directly to DVD-RAM. After the operation, the data can be written either to a DVD or CD or to a USB storage medium, or they are saved on a network server via NetBEUI, FTP or DICOM protocol (only with Aesculap DICOM Upgrade Set PV942).

The data saved locally on the hard disk should be deleted as soon as they have been saved externally, as Eddy DVD is not designed for permanent data storage.

The integrated patient data management system allows reliable attribution of stills and video sequences to individual patients.

The system is operated through the Eddy DVD documentation software (Eddy = Endoscopic Digital Documentation sYstem), which was specially developed for the requirements and procedures in the operating room.

3. Preparation and setup

Non-compliance with these rules will result in complete exclusion of liability on the part of Aesculap.

- When setting up and operating the product, always observe:
 - national regulations for installation and operation, and
 - national regulations on fire and explosion protection.

Note

For the safety of patients and users it is essential that the mains power cord and, especially, the protective ground connection are intact. In many cases defective or missing protective ground connections are not discovered immediately.

- Connect the device via the potential equalization terminal at the rear panel of the device to the potential equalization system of the medical premises.

Note

The potential equalization lead can be ordered from the manufacturer as art. no. GK535 (4 m length) or TA008205 (0.8 m length).

3.1 First use

Risk of injury and/or product malfunction due to incorrect operation of the electromedical system!

- Observe the instructions for use of all medical devices.

Positioning the Eddy DVD

- Place Eddy DVD on a level surface.
- Leave a distance of at least 5 cm to other devices (especially to HF-surgery devices) and items.
- Position Eddy DVD as close of possible between the video camera system and the monitor.

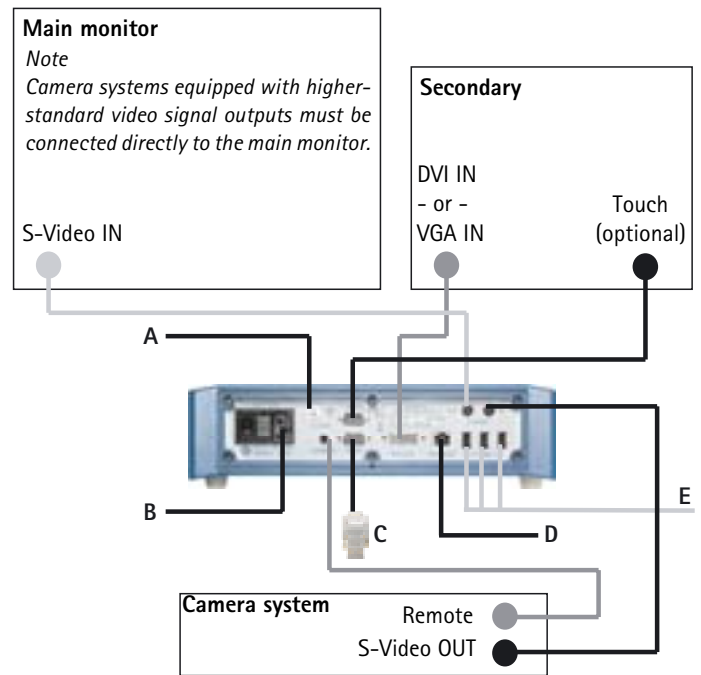
Fig. 1 Connection diagram

Legend

- A Potential equalization (building electrical installation)
- B Power supply (building electrical installation)
- C Dongle PV942 (optional)
- D Ethernet (building installation) (optional)
- E USB devices (optional)
 - Mouse
 - Printer
 - Keyboard
 - USB hard drive

Connecting the camera control unit

Note

For optimum video quality, only use shielded cables (see Chapter Accessories and replacement parts).

- Connect S-VIDEO output of the camera control unit with S-VIDEO IN input **20** of the Eddy DVD.
- Connect the remote control cable (see Chapter Accessories and replacement parts) to the Remote connector of the camera control unit and REMOTE connector **14** of the Eddy DVD.

Connecting the monitor or touch-screen monitor to operate the software

The documentation software of the Eddy DVD is operated via a monitor with DVI or VGA connection.

- For monitors with DVI input:
Connect video output DVI-I OUT **17** of the Eddy DVD to the DVI input of the monitor, using a DVI-D cable.

Note

The monitor must be connected and switched on before starting Eddy DVD.

- For monitors with VGA input:
Use a DVI-VGA adapter and an appropriate cable (both not included in the delivery scope of the system) for the connection between video output DVI-I OUT **17** of the Eddy DVD and a VGA video input socket of the monitor.
- For monitors with touch-screen function:
Connect TOUCH connector **15** of the Eddy DVD to the RS232-interface of the monitor cable, see Chapter Accessories and replacement parts.

Connecting a second monitor for live image display (optional)

The live image can be displayed on an optional second monitor equipped for S-VIDEO input.

- Connect S-VIDEO OUT **19** of the Eddy DVD with the S-VIDEO input socket of the monitors cable, see Chapter Accessories and replacement parts.

Connecting the mouse

All functions of the Eddy DVD can be comfortably controlled through a touch-screen monitor. In setups without a touch-screen monitor, all functions can be selected with the mouse.

In any case, Aesculap recommends connecting the mouse as an additional input device.

- Connect the USB mouse at a USB socket **21** of the Eddy DVD.

Connecting an external keyboard (optional)

All functions of the Eddy DVD can be comfortably controlled through a touch-screen monitor. In setups without a touch-screen monitor, an external keyboard can be used as an alternative input device.

- Connect the USB keyboard at a USB socket **21** of the Eddy DVD.

Connecting a microphone (optional)

As another option, to record sound accompanying video recording, a microphone with a 3.5-mm jack connector can be connected to the Eddy DVD.

- Plug in the microphone at microphone socket **9** in the front panel of the Eddy DVD.

Connecting a printer (optional)

The device "USB printer" is not recognized by Eddy DVD!

- **Prior to connecting the USB printer to the Eddy DVD, install the appropriate printer driver software.**

There is also the option to print reports directly on any USB printer connected to the Eddy DVD.

- Install the printer driver, see Chapter Menu "Settings/Printer/Network/System".
- To connect the USB printer to the Eddy DVD: Connect the USB printer cable at the USB socket of the printer.
- Connect the USB printer cable at a USB socket **21** of the Eddy DVD.
- To connect the external power supply to the printer: Connect the mains power cord at the external power supply and plug it at a building mains power socket.

Connecting the mains power cord and equipotentialization cable

The mains voltage must correspond to the voltage indicated on the type plate at the back of the unit.

- Plug in the power cord, see Chapter Accessories and replacement parts at power connector **12** of the Eddy DVD.
- Connect the mains plug of the power cord to the building mains.
- Plug in the equipotentialization cable at equipotentialization connector **13** of the Eddy DVD.
- Connect the plug of the equipotentialization cable to the building mains equipotentialization connector.

Stacking of units

- Make certain the system is set up on a sufficiently stable support (e.g. a table, ceiling support, equipment cart, etc.).

Eddy DVD digital documentation system PV820

Connecting the accessories

Risk of injury due to unapproved configuration using additional components!

- For all applied components, ensure that their classification (e.g. Type BF or Type CF) matches the classification of the respective device.

Combinations of accessories that are not mentioned in the present instructions for use may only be employed if they are specifically intended for the respective application, and if they do not compromise the performance and safety characteristics of the products.

Any equipment connected at the interfaces must demonstrably meet the respective IEC standards (e.g. IEC 60950 for data processing equipment, IEC/DIN EN 60601-1 for medical electrical equipment).

All configurations must comply with system standard IEC/DIN EN 60601-1-1. The person connecting the devices is responsible for the configuration and must ensure that it complies with system standard IEC/DIN EN 60601-1-1 or equivalent national standards.

- Please contact your B. Braun/Aesculap Partner or Aesculap Technical Service, see Chapter Technical Service with any inquiries in this respect.

Connecting the power supply

Risk of death by electric shock!

- Only ever connect the product to power mains with equipment grounding conductor.

3.2 Software configuration in menu "Settings"

Once the Eddy DVD has been positioned and connected, the software has to be configured through the "Settings" menu.

- Have the general settings configured by trained personnel.
- Have the network and system settings configured by the system administrator.

Setting up a network connection

If data are to be saved through the network in a shared folder on a specified computer, the system administrator can link Eddy DVD to a network.

- Connect the network cable between NETWORK connector 18 and building network socket.
- Create and make shareable a folder for saving the Eddy DVD data on the selected computer in the network.

Switching on the Eddy DVD system and configuring the software

- Switch on the monitor.
- Switch on the Eddy DVD at On/Standby switch 1.
As soon as Eddy DVD is ready, the start page is displayed:

- Select "Settings" button.
Menu "Settings/General" is displayed.

Note

The following is a detailed description of the individual menus and possible settings.

Menu "Settings/General"

Hospital

Enter hospital name. Automatically displayed preselection; automatically allocated to all patient data.

Logo

Inserts individual logo to be printed in the top right corner of every report.

The logo must exist as a bitmap file with the file name "Logo.bmp" in the root directory of a USB device or a CD.

Convert the logo to a suitable format

(width: height > 3.6 or height < 240 pixels

Plug in the USB device holding the logo or insert the CD holding the logo in the Eddy DVD.

Press button "Import logo".

The message "Logo copy successful" will be displayed.

Format of date

Select format of date:

- DD.MM.YYYY= European format

- MM/DD/YYYY= US format

DD =Day in two-digit numerical representation

MM =Month in two-digit numerical representation

YYYY =Year in four-digit numerical representation

Date/Time

Enter present date and time.

Language

Select language:

German, English, French, Italian, Spanish, Czech

Save/

Start page

Save settings and switch to start page.

Reject/

Start page

Switch to start page without saving.

Eddy DVD digital documentation system PV820

Menu "Settings/Video"

- Image format**
 - Select image format.
 - BMP + JPEG
to save images in BMP and JPEG format.
 - JPEG
to save images in JPEG format only.
 - JPEG = high quality, compressed
 - BMP = highest quality, not compressed
- Video quality**
 - Select video quality/Compression rate:
 - High Quality = approx. 1 h video per DVD
 - Standard = approx. 2 h video per DVD
 - Long Play = approx. 3 h video per DVD
- Save/Start page**
 - Save settings and switch to start page.
- Reject/Start page**
 - Switch to start page without saving.

Menu "Settings/Camera"

Note
For the "Image and video" function, the camera must support the time-sensitive remote control function (not supported e.g. by camera PV420/PV422).

- Remote button function**
 - Select image or video function:
 - Image (still)
 - Video (Start/Stop video recording)
 - Sound on (acoustic signal)
 - with a remote control cable connected, via the remote control button on the camera head.
 - Image and video:
 - Press camera remote control button briefly (< 2 s):
Still image is saved
 - Press camera remote control button any longer (> 2 s):
Video recording is started or stopped.
 - Individual images can still be captured during video recording (by briefly (< 2 s) pressing the camera remote control button).

Menu "Settings/Printer/Network/System"

The menu "Settings/Printer/Network/System" is accessible only for authorized password holders.

The system is supplied with the password preset to "0000".

The system administrator must change the password in the "Settings/System" menu.

Password

Password

Enter password for access to menu "Settings/Network".

If the password was entered wrongly more than twice, the menu "Settings/General" will be displayed again.

OK

Confirm password.

Wrong password

If the wrong password was entered, "Wrong password" will be displayed under the input field.

Save/Start page

Save settings and switch to start page.

Reject/Start page

Switch to start page without saving.

Eddy DVD digital documentation system PV820

Menu "Settings/Printer"

Note
The menu "Settings/Printer" is only accessible to authorized password holders. System printer settings should be changed only by specialist IT personnel.

Open System-Printer-Settings

Opens the Windows printer system control panel

Install printer driver

Opens the "Install printer driver" menu

Menu "Install printer driver"

Install printer driver

The selection list shows the printer drivers found in the system. These drivers may not be installed yet. To install a printer driver, select the driver and click on "Continue" button.

Installation from CD

Other printers can be installed using the appropriate printer driver CD.

Menu "Settings/Network/General"

IP address

Get IP address automatically (DHCP).
- or -
Without DHCP: Enter IP address for Eddy DVD.

Advanced

Enter advanced network settings: Select "Advanced" and proceed as described below.

Save/Start page

Save settings and switch to start page.

Reject/Start page

Switch to start page without saving.

Menu "Settings/Network/General/Advanced"

Gateway

Enter network gateway (if necessary).

Subnet mask

Enter subnet mask (if necessary).

Primary DNS

Enter primary Domain Name Server (DNS), if necessary.

Secondary DNS

Enter secondary Domain Name Server (DNS), if necessary.

Save/Start page

Save settings and switch to start page.

Reject/Start page

Switch to start page without saving.

Eddy DVD digital documentation system PV820

Menu "Settings/Network/NetBEUI"

- Network drive** Enter shared folder on a computer in the network in which Eddy DVD will save its data (e.g. \\server\share).
- Network username** Enter the user authorized to access the network drive (if necessary).
- Network password** Enter password for the user who is authorized to access the network drive.
- Network domain** Enter the network domain to which Eddy DVD belongs (if necessary).
- Save/Start page** Save settings and switch to start page.
- Reject/Start page** Switch to start page without saving.

Menu "Settings/Network/DICOM"

Note
For DICOM settings and functions refer to TA011713, "Instruction for use of DICOM Upgrade Set PV942".

Menu "Settings/Network/FTP"

- FTP server and port** Enter the name or IP-address and the port of the FTP server.
- Remote directory** Enter the directory on the FTP server.
- FTP user name** Enter FTP user name.
- FTP password** Enter FTP password.
- Passive mode** Select whether passive mode (PASV) is to be used.
- Save/Start page** Save settings and switch to start page.
- Reject/Start page** Switch to start page without saving.

Menu "Settings/Network/Test"

- Test NetBEUI connection** Eddy DVD attempts to establish a connection to the configured network drive.
- Test DICOM connection (Echo request)** Eddy DVD attempts to connect to the configured PACS server and to send a message (Verification SOP Class).
- Test FTP connection** Eddy DVD attempts establishing an FTP connection.
- Save/Start page** Save settings and switch to start page.
- Reject/Start page** Switch to start page without saving.

Menu "Settings/System"

- Change password** Enter a new password.
- Repeat password** Repeat (confirm) entry of the new password.
- Calibrate touch screen** If the touch screen requires adjustment: Carry out touch screen calibration, see Chapter Calibrating the touch-screen monitor.
- Write log files to DVD/USB** All log files are written to a DVD/CD or to an external USB device, to the folder EDDY/logs, to be analyzed for troubleshooting in case of problems.
- Version** In case of any software problems or to change to a newer version: Read the software version in use.
- Save/Start page** Save settings and switch to start page.
- Reject/Start page** Switch to start page without saving.

Eddy DVD digital documentation system PV820

Calibrating the touch-screen monitor

Touch the target areas on the touch-screen monitor with a finger or a selection pen. (25 targets).

Touch the four small target areas in the corners of the screen.

Testing the touch-screen monitor

Touch the touch screen with a finger or a selection pen. The mouse pointer should jump to the point touched on the screen.

If the calibration is correct:

Press .

If the calibration is incorrect:

Press  and repeat the calibration procedure.

4. Working with Eddy DVD PV820

4.1 System set-up

- Switch on the camera system.
- Switch on the monitor.
- Verify that the DVI video input selected on the monitor is the same as the one that is actually used by Eddy DVD.
- If data are to be read in from the network: Make certain that the network cable is connected to NETWORK connector **18** and that the network connection is operational.
- Make certain that the mouse (and the external keyboard, if applicable) is connected correctly.
- Press On/Standby switch **1** of the Eddy DVD.
On/Standby LED **2** lights up.
Eddy DVD is starting up. The start procedure will take approx. 60 s.
Eddy DVD is ready as soon as the following start page is displayed.

- Open the respective menus via the buttons "New patient", "Find patient" on the navigation bar at the lower edge of the screen, see Chapter Safe operation.

4.2 Function checks

- Check that the Eddy DVD start page is displayed.
- Check that Eddy DVD responds to the input devices (touch-screen monitor or mouse and keyboard, if connected).
- Having entered the data of the first patient, make certain that the camera image is displayed in menu "live image".
Alternatively, call up a "test patient" for the purpose of this check.

4.3 Safe operation

Eddy DVD offers the following control functions:

- Creating a new patient
- Finding patient data
- Recording still images, video sequences and sound
- Managing data recorded on DVD/CD or USB storage devices, or through the network
- Printing a report
- To change the settings, see Chapter Software configuration in menu "Settings"

CAUTION

Damage to the touch-screen monitor caused by touching it with a sharp object (e.g. a pencil or a ballpoint)!

- Touch the touch screen with a finger or a selection pen only.

- To select a function, click on the respective button or input field
 - with the left mouse button.
 - or –
 - touch the respective field on the touch screen with a finger or a selection pen.

Note

All control functions are accessible through clicking with the left mouse button or through touching the touch-screen monitor, if such a monitor is part of the setup.

The system can also be operated through a combination of the input devices mouse, touch-screen monitor and keyboard.

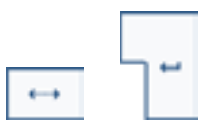
Eddy DVD digital documentation system PV820

Explanation of the control elements on the screen

The control elements on the screen are explained through the following examples.

Keys and buttons

	Navigation bar
Contains buttons for selecting the main functions	

	"Tab" key, "Return" key
Places the cursor into the next input field In the "Notes" field, the "Return" key produces a line break.	

	"Alt" key
Switches the keyboard display on the screen to special characters After selecting a special character, the keyboard switches back to standard characters	

	"Up" and "Down" keys
Opens and closes multi-line fields or scrolls up or down within such fields	

	Light gray button
Indicates that the corresponding function is inactive (in the present operating situation)	

Fields

	Blue data field
This input field must be filled out, otherwise a message will be displayed (compulsory input)	

	White data field
This input field can be filled out if required (e.g. Notes)	

	Gray info field
Shows information (e.g. DD.MM.YYYY), but cannot be edited	

	White input field
For data input via the keyboard; the cursor must be in the input field so that data can be entered	

	Light blue input field
Indicates the active input field with the cursor in it	

Inserting a DVD/CD

Loss of data saved on DVD/CD!

- Only use clean and dry DVDs/CDs.
- Do not touch, stain or scratch the recording side.
- Do not stick labels on DVD/CD. Only inscribe with a soft fiber pen.
- If necessary, clean the recording side with a soft dry tissue.
- Store CDs and DVDs in a CD-pouch or protective sleeve.

- Press EJECT button 6.
The DVD/CD inserted in DVD/CD drive 5 is ejected.
- Insert the DVD/CD, with the print side up, into DVD/CD drive 5 until the DVD/CD is automatically pulled into the drive.
With DVD/CD fully inserted, it will still take a few moments for Eddy DVD to read and process the disk. During this period, Read/Write-LED 7 will keep flashing. Only after this, the DVD/CD is ready for data reading or writing.

Creating a new patient

- Select button "New patient"  on the start page.
Menu "New patient" is displayed.
- Enter the patient data as described below.

Menu "New patient"

ID number (mandatory input)	Enter the patient's ID number. (without special characters \, /, *, ", <, >, ?, !).
Name (mandatory input)	Enter the patient's surname.
First name	Enter the patient's first name.
Date of birth	Enter the patient's date of birth. The set format of date is displayed in the gray info field.
Surgeon	Enter the surgeon's name. - or - Open gray info field. A selection list of the 15 surgeons that were entered most recently is displayed. To change the selection list, press button  , see menu "Surgeon / Operation".
Operation	Enter the kind of operation. - or - Open gray info field. A selection list of the 15 operations that were entered most recently is displayed. To change the selection list, press button  , see menu "Surgeon / Operation".
Notes	Enter notes if desired. More lines for notes can be created by pressing the Return key.
Start page	Save the entered data and switch to the start page.
Save/ New patient	Save entered data and switch to new, empty menu "New patient".
Save/Live image	Save entered data. The "Live image" menu for the selected patient is displayed.
Data incomplete	Data will only be saved if all (blue) compulsory fields were filled out correctly. Click on Complete data and complete data in menu "New patient". - or - Reject data.
ID number already exists	Data cannot be saved if the entered ID number has already been allocated to another patient. Enter unique ID number and confirm with "OK".

Possible messages

Eddy DVD digital documentation system PV820

Creating a new surgeon or new operation

- Select the appropriate button  in menu "New patient" or menu "Change patient".
The menu "Surgeon / Operation" is displayed.
- Enter the new surgeon or new operation as described below.

Menu "Surgeon / Operation"

New entry

Enter new surgeon or operation and press button .

Up/Down

Select surgeon or operation and move it to the intended position by pressing button  or .

Delete

Select surgeon or operation and delete the selected entry by pressing button .

Finding patient data

If videos, images or patient data from a previous session with Eddy DVD have been saved on the inserted DVD/CD, the connected USB storage device or the network, such data can be read in via the "Find patient" menu.

Note

To access data from the network, the network connection must have been established before Eddy DVD was started.

When data are to be read from a DVD/CD, it can take a few moments after inserting the DVD/CD until Eddy DVD has read in and processed the data. During this period, Read/Write-LED 7 will keep flashing.

- Ensure that the correct DVD/CD was inserted or the required USB storage device is connected and/or the network connection is up.
- Select button "Find patient"  on the start page.
The menu "Find patient" is displayed.
- Find patient data as described below.

Menu "Find patient"

Select the register data medium on which the patient data are to be searched.

- HD** = Hard disk
- DVD** = DVD or CD (DVD/CD must be in the DVD/CD drive)
- USB** = External USB storage device
(USB storage device must be connected)

The following registers are available only if they have been configured accordingly in the Settings/Network menu:

- LAN** = Network NetBEUI protocol
- FTP** = Network FTP protocol
- PACS** = Network DICOM protocol (only with Aesculap DICOM Upgrade Set PV942)

Search for Position the cursor in the selected input field.
Name Enter complete name or ID.
ID - or -
 Start entering letters or numbers.
 Patients found matching the entered data are displayed in the selection list.

Date Enter time interval "from - to" and press button .

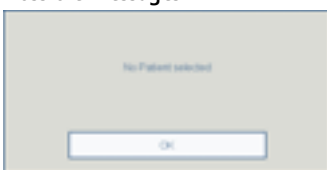
Start page Switch to the start page.

Live image Call up the "Live image" menu for the selected patient. *This function is active only if one individual patient was selected in the patient list to the right.*

Data overview Call up the menu "Data overview" for one patient or several patients selected.
The menu "Data overview for one patient" or "Data overview for several patients" is displayed.

No patient selected If no patient was selected, the menus "Live image" and "Data overview" cannot be called up.

Possible messages

Eddy DVD digital documentation system PV820

Recording still images, video sequences and sound

- Create or select a patient via the button "New patient"  or "Find patient"  on the start page.
The patient data of the selected patient are displayed.
Other buttons are displayed in the navigation bar.
- Select "Live image" button .
- Use the menu "Live image" for recording still images, video sequences and sound as described below.

Note

With a remote control cable connected and configured appropriately, see Chapter Software configuration in menu "Settings", still images and video sequences can also be recorded using the remote control button on the camera head.

*Video sequences can also be recorded using START/STOP button **3** on the Eddy DVD.*

Video sequences are automatically stopped as soon as the storage capacity of the hard disk or DVD is used up or the file reaches the maximum size for a DVD. Before the end of the recording time, messages indicating the remaining recording time will be displayed, "Time left 15 min/10 min/5 min/3 min/2 min/1 min". When the recording time is over, the message "Stopped recording" is displayed.

If the video sequence is recorded to hard disk HD, a new recording is started automatically.

Menu "Live image"

Storage location

Selection of the medium on which the data are to be saved:

HDD = hard disk

DVD = DVD-RAM

Note

Direct (live) storage is only possible on DVD-RAM media, which cannot be changed during recording. While a video sequence is recorded, the other button is deactivated.

The storage space used up on the medium is indicated by a bar display:

green: 0–75 % full

yellow: 75–90 % full

red: 90–100 % full

Image

Save image. The live image stream is interrupted and the recorded still image is displayed for approx. 1 s.

Start video

Start recording of a video sequence. While recording, a red dot is displayed in the upper left corner of the screen. The button changes to "Stop video". The Video START/STOP-LED 4 at the Eddy DVD is on.

Stop video

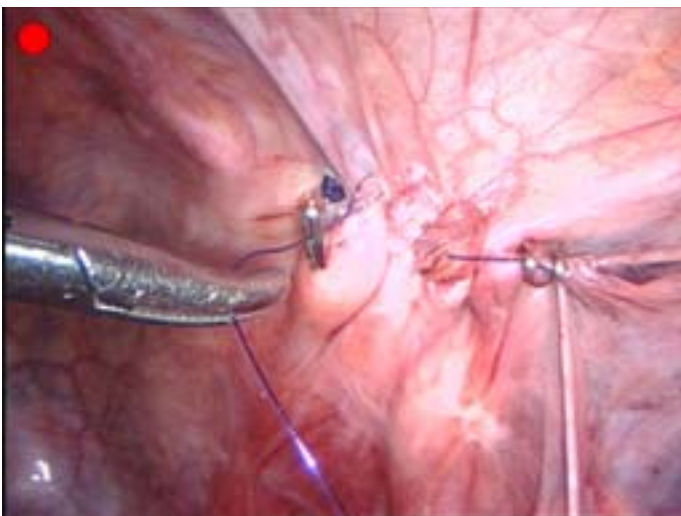
Stop recording of a video sequence. The red dot disappears. The button changes to "Start video". The Video START/STOP-LED 4 at the Eddy DVD goes off.

Data overview

The "Data overview" menu for the selected patient is displayed.

"Full image" view

To display full-screen images (as "Full image"): Click on image.
Back to overview display:
Click on image again.

Eddy DVD digital documentation system PV820

Managing image data and changing patient data

Managing recorded image data

Recorded images are cached by Eddy DVD and can be viewed, selected or deleted in menu "Data overview" prior to saving on DVD/CD or USB storage device or through the network.

Note

Still images and video sequences cannot be edited on the Eddy DVD.

Note

"Save on network" is only possible if the network connection is up and the system administrator has released a network drive for sharing, see Chapter Software configuration in menu "Settings".

"Write DVD/USB" is only possible if a writable DVD or CD has been inserted or a USB storage device connected and if there is sufficient free storage space on the respective media/device.

Changing patient data

The data that were entered for a patient in menu "New patient" can be changed in menu "Data overview/Change patient data".

Calling up the "Data overview" menu

➤ Call up the patient data via the menu "Find patient" and switch to menu "Data overview" by actuating the button "Data overview"  in the navigation bar.

- or -

Switch directly from menu "Live image" to menu "Data overview", using button "Data overview".

An overview of the data of the selected patient and the available preview images are displayed.

Other buttons are displayed in the navigation bar.

➤ View, save or delete data as described below.

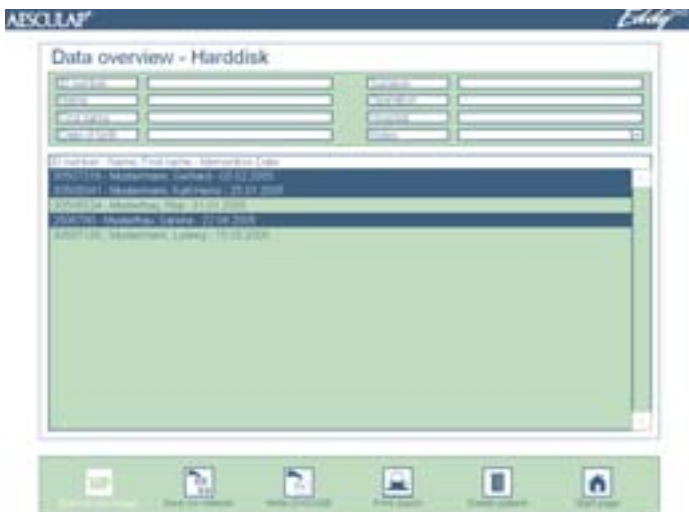
CAUTION

Unrecoverable deleted patient data, still images and/or video sequences!

➤ Before clicking on the appropriate "Delete" icon, save data that are still required to a DVD/CD, a USB storage device or to the network.

Menu "Data overview" (for several patients)

Data overview

In Data overview, individual patients can be selected. The active storage medium is displayed in the header:
 - Hard disk
 - DVD
 - USB
 - Network

Navigation bar

The buttons in the navigation bar allow copying of patient data to DVD/CD, USB or network, as well as printing or deleting of the data.

Note

The above patient data fields are left empty in this overview display.

Menu "Data overview – hard disk" (for one individual patient)

Menu "Change patient data"

Changing patient data

Switch to submenu "Change patient data" by clicking on the patient data and change data as required. (The ID-number in the grey info field cannot be changed.) After completion of the changes click on button "Back to data overview".

Images (Z)/Videos (Z) (index cards)

Overview of all still images/video sequences that have been recorded for the selected patient. The figure in brackets (Z) represents the number of recordings.

Select images on index card "Images (Z)"

Select one or several images by clicking on them in the overview. The selected images will be displayed in blue frames. To deselect, click again on the framed image.

Select videos on index card "Videos (Z)"

Click on a video in the overview. (If two videos are selected at the same time, no video can be played; the „Play" icon will be gray.)

Comments

Adding a comment to the selected image

View on index card "Images (Z)"

The selected image is displayed in full-screen mode, as a "Full image".

Play on index card "Videos (Z)"

The selected video is played.

Delete image(s) on index card "Image(s) (Z)"

All images/videos in blue frames selected in the overview will be deleted.

Delete video(s) on index card "Videos (Z)"

Save on network

All data or the selected images and videos of the present patient are saved to the network.

Write DVD/USB

All data or the selected images and videos of the present patient are saved to DVD/CD or an external USB storage device.

Delete patient

The entire data set of the selected patient, including all associated still images and video sequences, is deleted.

Start page

Switch to the start page.

Eddy DVD digital documentation system PV820

View image (after clicking on "View" on index card "Images (Z)"): Menu "Data overview/View image"

Full image

To display full-screen image ("Full image"): Click on image.
Back to overview display: Click on image again.

Next picture

Jump to next image if several images have been selected.

Comments

Add comment to image.


Delete image

Delete image irrevocably.

Closing

Close individual image display and switch to "Data overview" menu.

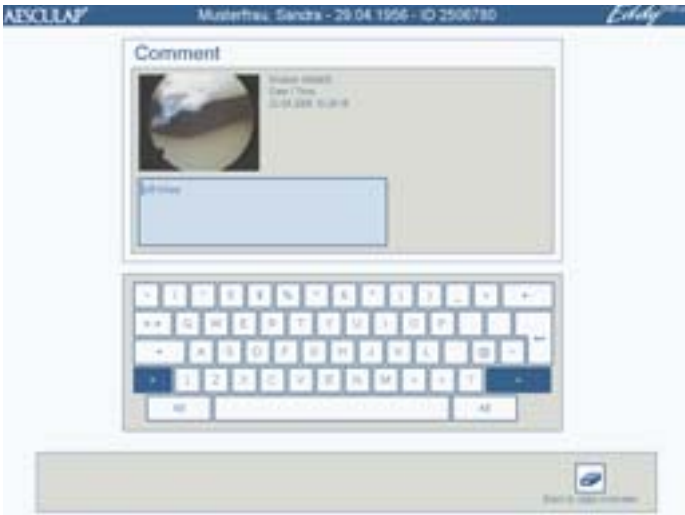Menu "Data overview/Comment"

Comments

Select an image from the overview by clicking on it. Enter up to three lines of comment per image. (If two images are selected at the same time, comment input is not possible; the "Comment" icon will be gray.)
Date and time of recording are displayed next to the image.

Full image

To display full-screen image ("Full image"): Click on image.
To return to comment view: Click on image again.

Play video sequence (after clicking on "Play" on index card "Videos (Z)":
Menu "Data overview/Video replay"

Full image

To display full-screen video ("Full image"):
Click on video.
Back to overview display:
Click on video again.

Progress bar

To forward or rewind the video to any position: Move blue position pointer.
Date and start time of the video recording are displayed in the lower left corner of the screen.
The actual position on, and the total running time of the video are displayed in the lower right corner.

Rewind

Rewind the video sequence to start of video.

Play

Start the video sequence.

Stop

Video sequence stops.

Image

Still image from present video is transferred to "Data overview/Images" menu.

Delete video

Delete video sequence irrevocably.

Close

Close video display and switch to "Data overview" menu.

Writing data to DVD/CD or USB storage device

Risk of data loss on the USB storage device!

- Deactivate the USB storage device before removing the USB storage device.

Note

The USB storage device is deactivated automatically when shutting down the Eddy DVD.

All data or the selected images and videos of the present patient are saved to DVD/CD or an external USB storage device.

- Click on **Write DVD/USB** in the Data overview screen.

The following selection window will be displayed:

The following options can be selected:

- Target drive
 - DVD or CD
 - USB storage device
(Options only active if a DVD/CD is in the drive or a USB storage device is connected.)
- Selected data
 - Selected data (selected images and videos)
 - All data

Data saving is started by pressing the **Write DVD/USB** button.

The USB device can be deactivated without data saving by pressing the **Deactivate USB device** button.

Eddy DVD digital documentation system PV820

The following choice is prompted when saving data to a USB storage device:

- Yes: The USB device is deactivated after data saving and can be removed at that stage.
- No: The USB device will remain active for saving other data.

Other prompts

Patient already exists. Add data? Query displayed during save if the patient already exists on the target drive (DVD/CD, USB storage device or network drive). The new data will be appended to the existing data.

Remove archived data locally? Query displayed when data have been saved successfully on the network drive or on a DVD/CD or USB storage device. Aesculap recommends deleting data on the (local) hard disk as soon as they have been archived.

Data folder structure and file formats

Data are saved to DVD/CD, USB or network, in the following structure:

- The folder name corresponds to the Patient ID
- The file "index.html" is found in the root directory. Open this file with any HTML browser (JAVA version 1.5 or later required). The file allows easy managing and viewing of the contents of the folder through an HTML browser on any PC (see instructions for use of image viewer software PV943 for Eddy DVD, TA011486).
- The patient data are stored as ASCII-text in the file "patient.dat".
- The file names of the still images and video sequences are composed as follows:
"Date of recording/Time of recording/Patient ID/Sequential no./Format"
- Still images are stored in 2 formats:
 - uncompressed Bitmap format (*.BMP)
 - compressed JPG format (*.JPG)
- Video sequences are stored in MPEG2 format (*.MPG).
The file with the extension *.MPG.BMP is only used for the preview display in menu "Data overview".

Example:


Dateiname	Größe	Typ	Geändert
2004-09-20 12-57-27 0123456789 0.mpg	874.953 KB	MPEG-Daten	20.09.2004 15:46
2004-09-20 12-57-27 0123456789 0.mpg.bmp	1.216 KB	Bitmap	20.09.2004 12:57
2004-09-20 12-59-44 0123456789 0.bmp	1.216 KB	Bitmap	20.09.2004 12:59
2004-09-20 12-59-44 0123456789 0.jpg	9 KB	JPEG-Bild	20.09.2004 12:59
2004-09-20 13-00-37 0123456789 1.bmp	1.216 KB	Bitmap	20.09.2004 13:00
2004-09-20 13-00-37 0123456789 1.jpg	9 KB	JPEG-Bild	20.09.2004 13:00
2004-09-20 13-00-54 0123456789 1.mpg	891.300 KB	MPEG-Daten	20.09.2004 15:46
2004-09-20 13-00-54 0123456789 1.mpg.bmp	1.216 KB	Bitmap	20.09.2004 13:00
patient.dat	1 KB	DAT-Daten	20.09.2004 13:32

Eddy DVD digital documentation system PV820

Printing a report

A report of the complete set of data and images for the selected patient will be printed.

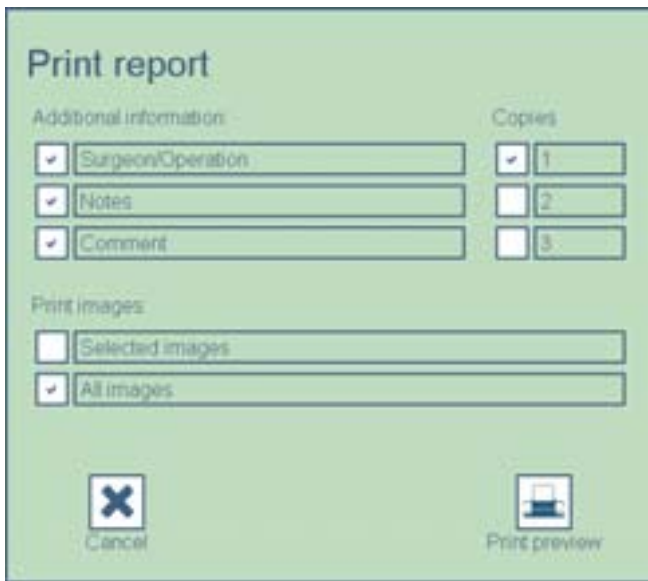- Click on **Print report** in the Data overview screen. The following selection window will be displayed:

The following options can be selected: (These options are only active if the respective data actually exist. Otherwise the respective option is displayed on a gray background.)

Additional information:

- Surgeon/Operation
- Notes
- Comment

Print images:

- Selected images
- All images

Copies:

- 1, 2 or 3

- Select the required options.
- Click on **Print preview**.

The following print preview is displayed:

Fig. 2 Example: Two-page report without comments

- Click on **Back** or **Forward** to scroll back or forward by one page in the report.
- Click on **Print report**. The print job is sent to the printer.

Operating the printer

- Follow the manufacturer's instructions for use.

4.4 Switching off the Eddy DVD

Data loss and software instability!

- Prior to switching off the Eddy DVD, shut down the software correctly, using the "Exit" button on the start page.
 - or -
 - Press the Standby switch in the front panel of the Eddy DVD.

- Select "Exit" button on the start page.
 - or -
 - Press the Standby switch in the front panel of the Eddy DVD. The software is shut down and the Eddy DVD switches off automatically (Standby mode).
- Once the On/Standby LED 2 has gone dark, press ON/OFF switch 10 at the Eddy DVD or switch off at the master switch of the equipment cart.

5. Validated processing procedure

Note

This product remains in the O.R. area and is not subject to the normal processing cycle.

Note

Adhere to national statutory regulations, national and international standards and directives, and local, clinical hygiene instructions for sterile processing.

Note

Up-to-date information on processing can be found on the Aesculap Extranet at www.aesculap-extra.net

Note

Successful processing of this medical product can only be ensured if processing is performed through a validated processing procedure. The user/processor is responsible for the validation.

5.1 General notes

Only use process chemicals that have been tested and approved (e.g. VAH/ DGHM or FDA approval or CE mark) and which are compatible with the product's materials according to the chemical manufacturers' recommendations. All process parameters specified by the chemical's manufacturer, such as temperatures, concentrations and exposure times, must be strictly observed. Failure to do so can result in the following problems:

- Material damage, e.g. corrosion, cracks, fracturing, premature aging or swelling.
- Do not use process chemicals that cause stress cracking or brittleness of plastic materials.

For further detailed advice on hygienically safe and material-/value-preserving reprocessing, see www.a-k-i.org

- Remove visible surgical residues as completely as possible, using a moist, lint-free wipe.

5.2 Manual cleaning/disinfecting

DANGER

Risk of electric shock and fire hazard!

- Unplug the unit before cleaning.
- Do not use flammable or explosive cleaning or disinfecting solutions.
- Make certain that no fluids will penetrate the product.

CAUTION

Damage to, or destruction of the product caused by mechanical cleaning/disinfecting!

- For this product, only apply manual cleaning/disinfecting by surface wipe disinfection.
- Do not spray surface cleaning/disinfecting agents into open sockets (e.g. USB sockets, Video input and output sockets, mains power socket) or into the DVD slot.
- Do not immerse or rinse the product in fluids under any circumstances.
- Do not sterilize the product under any circumstances.

CAUTION

Damage to the product due to inappropriate cleaning/disinfecting agents!

- Only use cleaning/disinfecting agents approved for surface cleaning. Follow the manufacturer's instructions for the respective cleaning/disinfecting agent.
- Observe specifications regarding concentration, temperature and exposure time.

Eddy DVD digital documentation system PV820

Wipe disinfection for electrical devices not to be sterilized

Stage	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Wipe disinfection	RT	≥1	-	-	Meliseptol HBV wipes 50 % propan-1-ol

RT: Room temperature

Stage I

- Remove any visible residues with a single-use disinfecting wipe.
- Wipe all surfaces of the optically clean product with a new, single-use disinfecting wipe.
- Observe the specified application time (1 minute minimum).
- After disinfection, flash off the product for at least 1 min.

5.3 Inspection, maintenance and checks

- Inspect the product after each cleaning and disinfecting cycle to be sure it is clean, functional and undamaged.
- Set aside the product if it is damaged.

6. Maintenance

To ensure reliable operation, the product must be serviced at least once a year.

This is equivalent to obligatory safety inspections according to §6 MPBetreiberV (Germany).

For services to that end, please contact your national B. Braun/Aesculap agency, see Chapter Technical Service.

7. Troubleshooting list

7.1 Messages on Eddy DVD

General messages

Message	Cause	Remedy
Memory space running low. Archive some data to release memory space	There is only little memory space left available	Archive some data on DVD/CD or network and delete these data from memory
No patient selected	Data cannot be displayed because no patient was selected	Click on a patient; the patient data will be displayed in the right column
One or several patient data input fields have not been filled out	Blue data fields must be filled out (compulsory input)	Fill out correctly all compulsory input fields
Data incomplete	Blue data fields must be filled out (compulsory input)	Fill out correctly all compulsory input fields
Inconsistent patient data ID-number	Patient data file error	Call up and check the respective patient data; amend or delete data as necessary
ID number already exists	The ID number is already in use on the hard disk	For the new patient a new ID number must be entered
The ID number must not contain special characters	The ID number contains invalid special characters such as \, /, *, ", <, >, ?,	Check the ID number and remove any special characters
Invalid date of birth	The date of birth was not entered in a valid format	Check and correct the date-of-birth entry
Video replay failure	Video was damaged (e.g. due to file system error)	Call Technical Service
Save to USB device unsuccessful	The data could not be saved to the USB device	Check USB device write-protection status Test USB device function with a PC
Deactivating of USB device unsuccessful	USB device could not be deactivated	Repeat deactivating Shut down Eddy DVD
No DVD/CD inserted	There is no disc in the DVD/CD-drive	Insert disc in the DVD/CD-drive
DVD/CD is full	The storage capacity of the DVD/CD is exhausted	Insert new disc in the DVD/CD-drive
Unknown or illegal medium	The disc is not recognized as a DVD/CD	Inspect disc (for dirt, scratches, correct positioning in the drive, etc.)
DVD/CD has been formatted with an incompatible file system	The DVD/CD already contains a file system from a previous write session. Eddy DVD cannot save data on this disc	Insert new disc in the DVD/CD-drive
Not enough free space on DVD/CD	The storage capacity on the DVD/CD is insufficient for the selected data	Insert new disc in the DVD/CD-drive Deselect video sequences or images if necessary
Timeout on drive access	DVD/CD-drive not ready	Repeat procedure after a short while (10 s) If error recurs, call Technical Service
Not enough memory for caching	Not enough cache memory available on the hard disk	Call Technical Service

Eddy DVD digital documentation system PV820

Message	Cause	Remedy
Medium information cannot be read	DVD/CD read or write process failed	Replace DVD/CD; if necessary, use PC to copy data to a new medium If error recurs, call Technical Service
Medium cannot be read	DVD/CD read or write process failed	Replace DVD/CD; if necessary, use PC to copy data to a new medium If error recurs, call Technical Service
Burn process failed	General error during DVD/CD burning	Replace DVD/CD and restart disc burning procedure If error recurs, call Technical Service
Medium/drive not ready	DVD/CD-drive not ready	Repeat procedure after a short while (10 s) If error persists: Call Technical Service
Passwords not identical	The repeat entry of the password was not identical with the initial entry when the password was changed	Repeat procedure: The second (confirming) entry for the password must be the same as the first entry
Logo file not found	No CD inserted or no logo file existing on CD	Insert a correct DVD/CD with logo file logo.bmp
Logo copying unsuccessful	Insufficient storage space, file system corrupt or hardware fault	Call Technical Service
Incorrect logo format	Logo too large or incorrect width-to-height ratio	Convert the logo to a suitable format (width: height > 3.6 or height < 240 pixels)

Network messages

Message	Cause	Remedy
No network connection / connection lost?	Connection to network lost	Check network connection and network settings; repeat procedure If error persists: Call network administrator
Connecting to network failed	Eddy DVD cannot establish connection to network	
Invalid network path specified	The specified network drive is invalid (e.g. server name cannot be found)	
Access to net drive denied / Wrong user name or password	Wrong user name or password for access to network drive	Check user name and password; repeat procedure If error persists: Call network administrator
FTP transfer unsuccessful	An error occurred in sending data via FTP	Check network connection
FTP connection could not be established	The connection to the FTP server could not be established	Check network settings
	Wrong user name or password	Check user name and password for FTP server
	Network problem	Check network connection
Preparation of patient data unsuccessful	Adding of patient data to existing patient data failed	Call Technical Service
No access to remote directory. Change directory in FTP settings	FTP directory invalid	Check network settings (FTP directory)
	No access privilege for the FTP directory	Check FTP server privileges
Passive mode not supported by server. Change FTP settings	The passive (PASV) mode was selected, but is not supported by the FTP server	Check network settings (FTP mode)
FTP session interrupted	Connection cut by FTP server	Check network connection

Message	Cause	Remedy
FTP server error message	An FTP server error was detected	Call the system administrator of the FTP server
Wrong user name for FTP-Server	User name not known to FTP server	Check network settings
Wrong password for FTP-Server	Wrong FTP server password entered	Check network settings
Logon to FTP server unsuccessful (check user name and password)	User name not known to FTP server	Check network settings
	Wrong FTP server password entered	Check network settings
FTP connection could not be established - server unknown	FTP server inaccessible	Check network connection
	Invalid FTP server name	Check network settings
	FTP server name cannot be resolved	Check network settings Check network connection
Timeout during FTP access	FTP server does not respond	Check network connection Call the system administrator of the FTP server
General FTP error	An unspecified error occurred during the FTP session	Check network settings Check network connection Call the system administrator of the FTP server

7.2 Other malfunctions

Malfunction	Cause	Remedy
No menu displayed	Monitor switched off	Switch on monitor; restart Eddy DVD, if necessary, making certain that the monitor is switched on during system run-up
	Wrong channel setting for the video input signal	Make certain that the channel is set to the video input signal which is actually used
	Mains not connected correctly	Make certain that all cables are connected correctly
	Driver problem	Call Technical Service
No live image	Camera switched off	Switch on camera; restart Eddy DVD, if necessary, making certain that the camera is switched on during system run-up
	Camera not connected correctly	Make certain that all cables are connected correctly Test camera function through direct video connection between camera and monitor
Eddy DVD does not start	Mains cord not connected correctly	Check whether ON/Standby LED 2 is on
		Make certain that the power cord is connected correctly
Touch-screen monitor does not work	Touch-screen cable not connected correctly	Make certain that the touch-screen cable is connected correctly
		Restart Eddy DVD, if necessary, making certain that the touch-screen monitor is on during system run-up
	No recognizable cause	Call Technical Service

Eddy DVD digital documentation system PV820

Malfunction	Cause	Remedy
USB mouse does not work/ USB keyboard does not work	Mouse/keyboard not connected correctly Driver problem	Make certain that the mouse/keyboard is correctly connected at the USB socket Restart Eddy DVD with the mouse/keyboard connected Call Technical Service
	Mouse/keyboard defective	Replace mouse/keyboard
Eddy DVD does not respond/does not respond anymore	Hardware fault/driver problem/software error	Shut down and restart Eddy DVD Call Technical Service if necessary
Eddy DVD crashes/shuts down Eddy DVD does not recognize external devices	Overheating (especially in high ambient temperatures)	Switch off Eddy DVD and let product cool down Make certain that there is enough space between Eddy DVD and other objects, especially at the rear of the unit
	Hardware fault/driver problem/software error	Shut down and restart Eddy DVD Call Technical Service and specify the error message, if applicable
Eddy DVD does not save data to the network	Network cable not connected correctly	Make certain that the network cable is connected correctly
	Incorrect network settings	Have network settings checked by the system administrator
	No/insufficient privilege for access to network	Have network drive released for sharing by the system administrator
	IP address conflict	Have IP address allocated automatically, if possible (DHCP, see Chapter Software configuration in menu "Settings") Have network settings checked by the system administrator
Eddy DVD does not save data on DVD/CD	Storage space on DVD/CD used up	Replace DVD/CD
	Wrong DVD/CD	Check DVD/CD (for supported DVDs/CDs see Chapter Technical Service)
	Defective DVD/CD	Replace medium
	Hardware fault	Call Technical Service
Eddy DVD does not save settings	Hardware or software error	Call Technical Service

7.3 Changing fuses

Risk of death by electric shock!

- Unplug the unit before changing the fuses!

Fig. 3 Changing fuses

Allowable fuses:

IEC 127, D 5 x 20 mm, T 1.0 A, breaking capacity L (35 A), 250 V;
art. no. TA021410

- Use a small screwdriver to release clip on the fuse holder 11.
- Extract fuse holder 11.
- Replace both fuses.
- Reinsert the fuse holder in such a way that it audibly snaps into place.

Note

If the fuses blow frequently, the product is defective and needs to be repaired, see Chapter Technical Service.

8. Technical Service

Risk of injury and/or malfunction!

- Do not modify the product.

- For service and repairs, please contact your national B. Braun/Aesculap agency.

Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

Service addresses

Aesculap Technischer Service
Am Aesculap-Platz
78532 Tuttlingen / Germany
Phone: +49 7461 95-1601
Fax: +49 7461 14-939
E-mail: ats@aesculap.de

Or in the US:

Aesculap Inc.
Attn. Aesculap Technical Services
615 Lambert Pointe Drive
Hazelwood, MO 63042
Aesculap Repair Hotline
Phone: +1 800 214-3392
Fax: +1 314 895-4420

Other service addresses can be obtained from the address indicated above.

Eddy DVD digital documentation system PV820

9. Accessories and replacement parts

DICOM Upgrade Set/image viewer software for Eddy DVD

Designation	Art. no.
DICOM Upgrade Set for Eddy DVD PV920/PV940	PV942
Image viewer software for Eddy DVD	PV943

Cable

Designation	Art. no.
S-VHS connecting cable, 2 m	PV963
Serial connecting cable 9-pin sub-D, e.g. as touch screen cable	PV428
Remote control cable, 1.7 m (cinch-jack)	PV967
Remote control cable, 2.0 m (jack-jack)	PV968

Power cords and special power cords

Designation	Art. no.
Germany, Denmark, France, Austria, Belgium, Spain, Netherlands, Portugal, Luxembourg, Italy, Sweden, Norway, Finland	
Length 5 m	TE 730
Length 1.5 m	TE 780
United Kingdom, Ireland, 5 m	TE 734
USA, Canada, Japan, 3.5 m	TE 735
Special power cord IEC 320, 1 m	TE 676
Special power cord IEC 320, 2.5 m	TE 736

Note

For a complete overview of Aesculap Endoscopy equipment and accessories, see the Endoscopy Online Catalog at www.endoscopy-catalog.com

Replacement parts

Designation	Art. no.
Optical mouse with USB connection	PV920200
DVI cable, 3m	PV820201
Fuse IEC 127, D 5 x 20 mm, T 1.0 A	TA021410

10. Technical specifications

Classification acc. to Directive 93/42/EEC

Art. no.	Designation	Class
PV820	Eddy DVD digital documentation system	I

Mains power Mains connection socket	100–240 VAC, 50–60 Hz IEC 320
Power consumption	50 VA
Display resolution	1024 x 768 pixels
Color system settings	PAL: 625 lines, 25 images, line interval scanning 2:1 NTSC: 525 lines, 30 images, line interval scanning 2:1
System	1 GHz Intel® processor RAM 512 MB Windows XP embedded installed on solid state disk
DVD/CD-drive (compatible media and speeds)	DVD-RAM (Read 5x/Write 5x), DVD-R/+R/+RW (Read 8x/Write 8x), DVD-RW (Read 8x/Write 6x), DVD-R/+R Dual Layer (Read 8x/Write 4x), CD-R/-RW (Read 24x/Write 24x),
Hard disk	150 GB
Network connection	Ethernet 10/100 Mbit/s RJ45
Video inputs and outputs	S-VIDEO IN: Mini DIN socket S-VIDEO OUT: Mini DIN socket DVI-I OUT
Interfaces	1 x Remote: CINCH socket 4 x USB 2.0 2 x RS232 9-pin sub-D plug
File formats	Still images: Bitmap (BMP)/JPEG (JPG) Video sequences: MPEG-2 (MPG)

Video qualities (with running times as guide values)	High Quality (HQ): equivalent to approx. 60 min/DVD Standard Play (SP): equivalent to approx. 120 min/DVD Long Play (LP): equivalent to approx. 180 min/DVD
Dimensions (W) x (H) x (D)	305 x 85 x 305 mm
Weight	4.9 kg
Protection type acc. to EN 60529	IP 20 D
Compliance	93/42/EEC, Class I
Conforming to standards	EN 60601-1, protection class 1, EN 60601-1-2 EN 62304

10.1 Ambient conditions

	Operation	Transport and storage
Temperature	10 % — 40 %	-10 % — 50 %
Relative humidity	30 % — 75 %	10 % — 90 %
Atmospheric pressure	700 hPa — 1 060 hPa	500 hPa — 1 060 hPa

11. Disposal

Note

The user institution is obliged to process the product before its disposal, see Chapter Validated processing procedure.

Adhere to national regulations when disposing of or recycling the product, its components and its packaging! The recycling pass can be downloaded from the Extranet as a PDF document under the respective article number. (The recycling pass includes disassembling instructions for the product, as well as information for proper disposal of components harmful to the environment.) Products carrying this symbol are subject to separate collection of electric and electronic devices. Within the European Union, disposal is taken care of by the manufacturer as a free-of-charge service.

- Please contact your national B. Braun/Aesculap agency if you have any questions concerning the disposal of the product, see Chapter Technical Service.

12. Distributor in the US/Contact in Canada for product information and complaints

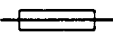
Aesculap Inc.
3773 Corporate Parkway
Center Valley, PA 18034
USA

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Aesculap AG