da Vinci[®] Xi[™] Instrument Arm Draping System User Manual Addendum

PN 553480-02 Rev. A 2015.07

Intuitive Surgical, Inc.
1266 Kifer Road
Sunnyvale, CA 94086 USA
www.intuitivesurgical.com

Intuitive Surgical, Sàrl
1, chemin des Mûriers,
1170 Aubonne Switzerland

0543

Copyright

© 2015 Intuitive Surgical, Inc. All rights reserved.

Trademarks

Intuitive, Intuitive Surgical, Beyond the Limits of the Human Hand, da Vinci, da Vinci S, da Vinci Si, da Vinci Xi, EndoWrist, TilePro and *InSite* are trademarks or registered trademarks of Intuitive Surgical, Inc. Other parties' trademarks are the property of their respective owners and should be treated as such.

Rx only

Contents

English 5

What to Do with This Document

Keep this addendum with the *da Vinci Xi* System.

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Introduction

This addendum to the *da Vinci Xi* System User Manual describes updated instructions specific to:

- *da Vinci Xi* Instrument Arm Draping Procedure

Keep this addendum with your *da Vinci Xi* System User Documentation.

Instrument Arm Draping Procedure

While draping the instrument arm, ensure that the drape pouch attached to the sterile adapter does not become pinched between the sterile adapter and the instrument carriage (Figure 1).

Figure 1 Pinched drape pouch (left), smooth drape pouch (right)

Special Considerations

If the *EndoWrist* Stapler is installed on an arm with a pinched drape pouch, a missing Reload message (Figure 2) may appear on the Vision Cart touchscreen. The purpose of the message is to ensure that a Reload is installed. If a Reload is installed correctly and the message persists, it may be caused by a pinched drape pouch.

Figure 2 Missing Reload message

To resolve the issue, verify the drape pouch around the sterile adapter is smooth:

1. Remove the sterile adapter.
2. Straighten the drape pouch.
3. Re-seat the sterile adapter.

For additional information on *EndoWrist* Stapler use, refer to the *da Vinci Xi EndoWrist* Stapler User Manual.

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