


1

- 1.1 Push the pads down to the lowest possible position.
- 1.2 Remove the screws in order: 1-2-3.


Replace screws no.2 with new ones supplied in the spare part bag.

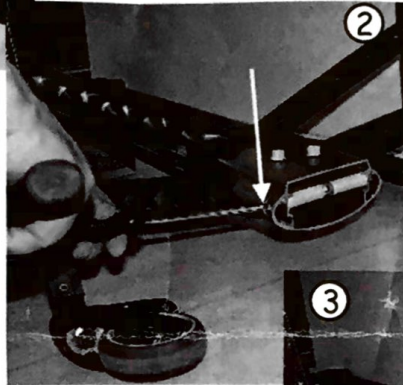


2

- 2.1 Lay down the walking table on the floor. Put something under the pillar to prevent tension between the pillar and the gas spring.
- 2.2 Remove the bottom cover and the 2 dummy plugs.
- 2.3 Knock out the bolt carefully.

- 3.1 Remove the release lever and pull out the gas spring from the bottom side. **Important!** Note how the lever is mounted before disassembling and ensure that the inner pillar does not come out!
- 3.2 Mount the new gas spring the same way.

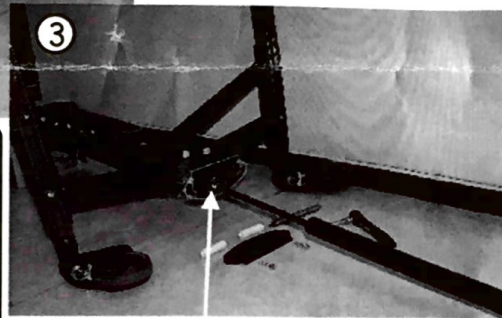


3

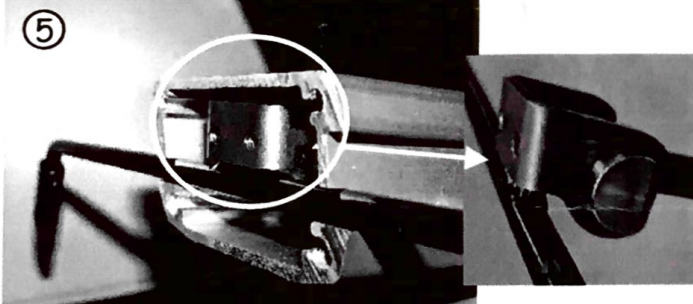


5

5. Mount the lever the same way as it was disassembled

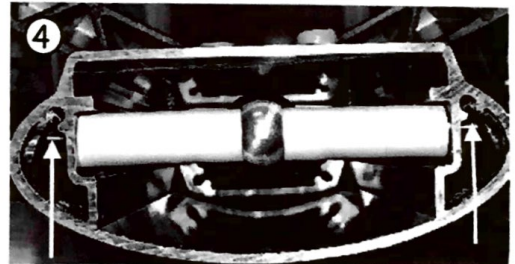

4

- 4.1 Mount the bolt and the 2 distance tubes. **NB!** The bolt must be centered so that the distance is equal on both sides.
- 4.2 Mount the bottom cover and the 2 dummy plugs

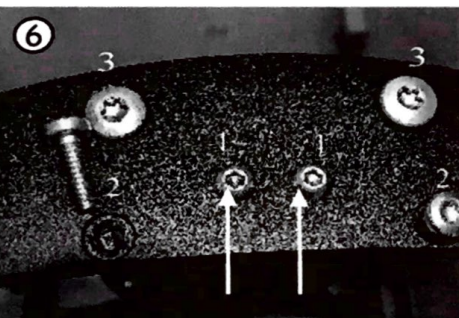

6

- 6.1 Insert all screws in order 1-2-3 (Partly in) Don't tighten until all are inserted.
- 6.2 Tighten all screws with a torque wrench.
  - Screw no. 1: 5 Nm
  - Screw no. 2: 16 Nm
  - Screw no. 3: 35 Nm



7

7. Check the operation before use

[www.svako.no](http://www.svako.no)

**svako easylift**

**KOZ2P03-450-1177-002 130N**

Nicht öffnen oder beschädigen - Hochdruck!  
 Do not open or damage - high pressure!  
 Ne pas ouvrir ou endommager - haute pression!  
 Non aprire o manomettere - alta pressione!

Made in Germany 15/09