CE

HANAULUX[®] *blue* 80 Hospital -valaisin

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Contents	Page
1 Notes on using these operating instructions	4
1.1 Explanation of symbols	4
1.1.1 Symbols used in the instructions.....	5
1.1.2 Symbols on the appliance	5
1.2 General safety information	6
1.3 Warranty	7
2 Operation	8
2.1 Checking the supply voltage	8
2.2 Mains and battery operation.....	8
2.2.1 Mains operation.....	8
2.2.2 Battery operation.....	8
2.3 Positioning the light head	10
2.4 System functions.....	10
2.5 Testing prior to every operation	12
2.6 3-monthly testing.....	13
3 Maintenance.....	14
3.1 Maintenance and repair	
3.2 Cleaning/Disinfection	14
3.3 Sterilising handles	15
3.3.1 Dismantling/assembly	15
3.3.2 Sterilisation.....	15
4 Repair	16
4.1 Changing the light bulb	16
4.2 Changing the battery	18
4.2.1 Removal	18
4.2.2 Fitting	18
4.3 Adjusting the brake force on the light head	20
4.4 Setting the spring resistance on the counterweight	20
4.5 Setting the height limit on the counterweight	22
4.6 Changing a fuse	22
4.7 Spare parts.....	22
5 Trouble shooting	25
6 Technical Data	27

Figures	Page
Fig. 1 Setting the supply voltage	9
Fig. 2 Mains and battery operation.....	9
Fig. 3 Positioning the light head	11
Fig. 4 Changing the light bulb	17
Fig. 5 Changing the battery	19
Fig. 6 Adjusting the brake force on the light head	21
Fig. 7 Setting the spring resistance on the counterweight	21
Fig. 8 Setting the height limit on the counterweight	23
Fig. 9 Changing a fuse	23

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1 Notes on using these operating instructions

Keep these operating instructions somewhere safe close to the appliance so that safety instructions and other important information can be consulted at any time.

Please read these operating instructions through carefully before initial operation of HANAULUX *blue* 80 Hospital. You will then know how to use the appliance to its full advantage and will avoid causing any damage.

Use these instructions to show operating personnel the functions and means of operating and maintaining the HANAULUX *blue* 80 Hospital lighting system.

We reserve the right to make technical improvements.

Maquet GmbH does not provide any warranty or accept any liability for the marketability and suitability of the appliance for any use except as indicated in the section „Intended use“ of the appliance.

With regard to translations into foreign languages, the German version of these operating instructions is mandatory.

Trademarks:

HANAULUX® and *blue* 80® are registered trademarks of Maquet GmbH.

All trademarks used in these operating instructions are the sole property of the manufacturer concerned.

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1.1 Explanation of symbols

Safety symbols draw your attention to operating errors affecting safety.

1.1.1 Symbols used in the instructions

COMMENT! is used if the correct functioning of the appliance may be affected.

ATTENTION! is used if the appliance may be damaged.

DANGER! is used if there is a risk of injury.

HOT SURFACE! warns of the risk of burns.

CE conformity mark

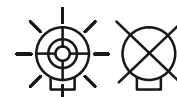
1.1.2 Symbols on the device

HOT SURFACE! warns of the risk of burns from hot surfaces.

ATTENTION! observe assembly and operating instructions.

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1.2 General safety information

Intended use:

- HANAULUX *blue* 80 Hospital is used to light the part of a patient being examined or operated on in clinics and doctors surgeries.
- The optimum examination distance is 70 to 130 cm.
- HANAULUX *blue* 80 Hospital is suitable for continuous operation when running off the mains.
- The batteries for HANAULUX *blue* 80 Hospital are used to supply power to the light in the event of a power failure.

Environmental conditions:

The following storage conditions apply to transport and storage times up to 15 weeks:

- Temperature: -25 to 70°C
- Relative humidity: 30-75%
- Air pressure: 700-1060 hPa
- Storage: in enclosed or sheltered spaces, otherwise the values under the location requirements apply.

Location requirements:

- HANAULUX *blue* 80 Hospital is not suited to use in hazardous locations.
- Maintain the ambient temperature of 10°C to 40°C whilst in operation.
- The relative humidity must not exceed 75%.

Operation:

- HANAULUX *blue* 80 Hospital is designed for distribution voltages of 115V/120V and 230V/240V. Depending on the supply voltage of the country of use, different fuse links must be used. Before first use check the set distribution voltage.

Maintenance:

- Please observe the special safety instructions in Section 3.

Repair:

- Please observe the special safety instructions in Section 4.

Disposal:

- Old appliances still contain valuable materials. Therefore, do not simply take old appliances to the nearest dump but contact your town/district council to find out about possible reuse.
- Batteries are to be taken away for raw materials treatment in accordance with local regulations and must not be disposed of with household waste.

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1.3 General safety notes

Requirements for safe operation

The safe and correct function of the HANAULUX *blue* 80 Hospital within the given technical data is only guaranteed if the following conditions are fulfilled.

- Extensions, modifications or repairs are carried out by Maquet or an authorized dealer.
- The HANAULUX *blue* 80 Hospital must be assembled in accordance with the enclosed installation instructions.
- Maquet original parts are used for servicing, repair and modification work as well as accessories.
- The HANAULUX *blue* 80 Hospital must be used in accordance with these operating instructions.

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Fig. 1

ATTENTION:

The HANAULUX blue 80 Hospital is designed for supply voltages of 115V/120V and 230V/240V. Depending on the line voltage of the country of use, different fuse links must be used. Before first use check the set supply voltage.

1. Remove the **appliance inlet (1)** below the **socket (2)**, the present supply voltage setting can be seen in the unit **window (3)**.
2. Check that **two fuses (4)** are fitted in accordance with Table 2-1e.

Table 2-1e Fuse links

No. off	Supply voltage	Value
2	115 V and 120 V	5,0 A
2	230 V and 240 V	2,5 A

3. To set the supply voltage, please see Section 4.5 Changing a fuse.
4. Insert the mains plug into the **socket (4)**.

2.2 Mains and battery operation Fig. 2

2.2.1 Mains operation

- During mains operation, LED 1 is green.
- When the batteries are being charged LED 2 flashes green.
- When the batteries are full LED 2 stays constantly green.

COMMENT :

During mains operation, the charging status indicator for the batteries LED 3 to LED 8 is **not** on.

2.2.2 Battery operation

In the event of a power failure, the light is supplied with current by the batteries. The batteries are then slowly discharged. LED 2 shows yellow.

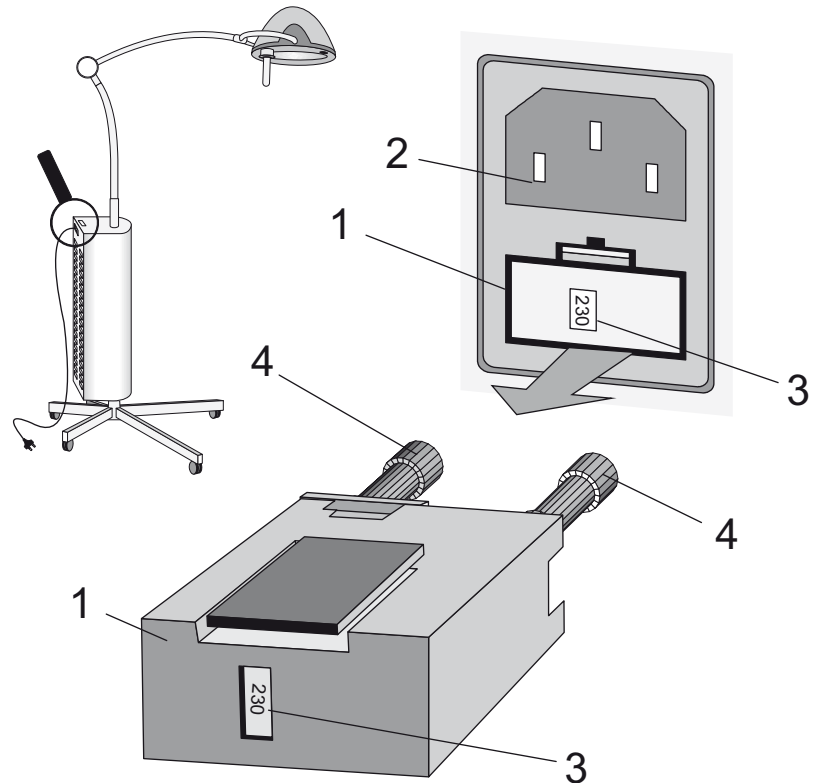
The charging status of the batteries is shown by LED 3 to LED 8. With decreasing battery voltage, the charging status indicated moves from + (PLUS) to - (MINUS). The indicator works in stages of approx. 0.5 V.

When the first red LED on the charging indicator lights up, a warning signal sounds. The light can only be operated for approximately another 15 minutes.


If the light is not switched off now the lamp will switch itself off at approx. 15V (total discharge protection).

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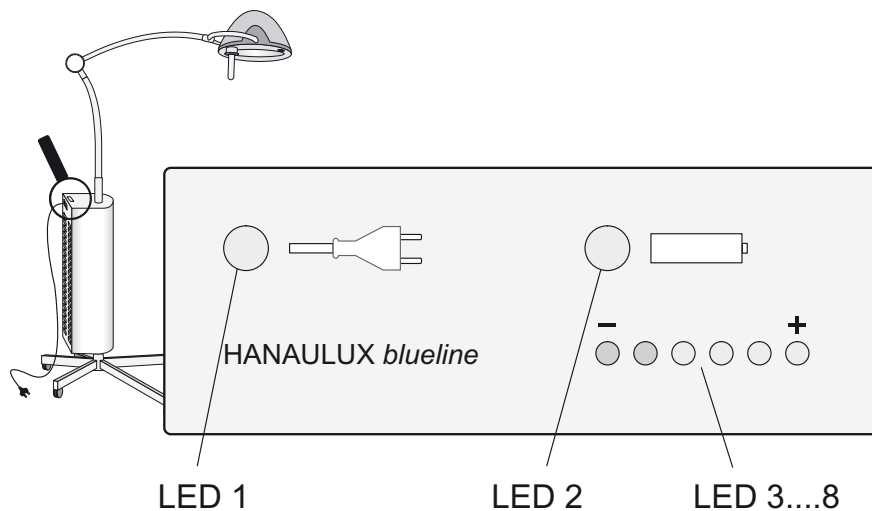2.3 Positioning the lighthouse *Fig. 3*

DANGER OF BURNS:

The glass cover of the lighthouse heats up during operation.

Do not touch the glass when adjusting the lighthouse.

DANGER of toppling:

Lack of attention when moving the light, especially over objects / electrical cables or uneven ground can cause the light to topple.

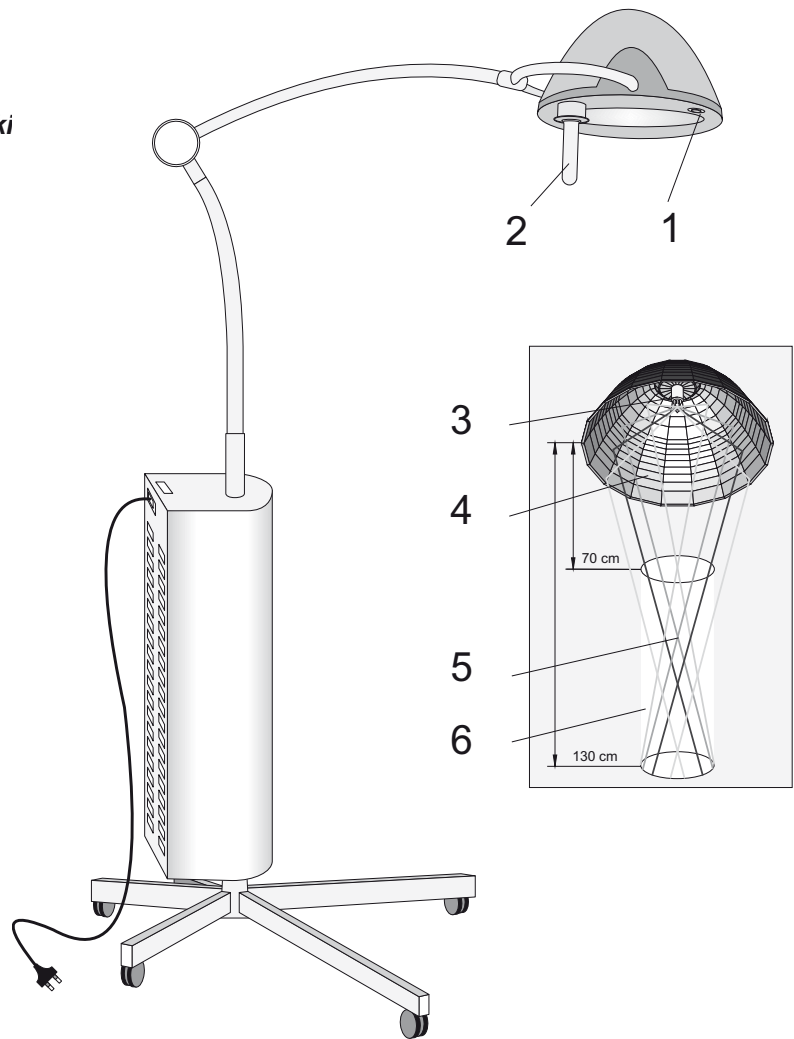
- Connect the mains plug to an earthed socket.
- Use **ON / OFF switch (1)** to switch the lighthouse on or off.
- Position the lighthouse using either the **sterilisable handle (2)**.
- Optimal working distance is 70 cm to 130 cm.

2.4 System functions *Fig. 3*

- The light source is a special type of **halogen bulb (3)** with specific filament position which is adjusted to the optical system.
- The light collected and concentrated in the **reflector (4)** produces a **cascade of light spots (5)** underneath the HANAULUX *blue 80* lighthouse.
- This generates an almost **cylindrical of light (6)** where the light is homogeneously distributed across the working range between 70 cm and 130 cm.
- A brilliant light of 80.000 lux is provided by the *blue 80* Hospital.

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2.5 Check before every use

NOTE:

To guarantee a faultless function in case of power-failure a check before every use must be done.

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Item	Point checked	Mains operation LED 1	Battery operation LED 2	Charging mode LED 3...8
1	Connected to mains voltage, Switch on light	Steady green light	Green blinking or steady green light	No display
2	Unplug mains (The lamp must stay on)	Goes out	Shines yellow	Battery operation. Green LED 8 must shine
3	Plug in mains again	Steady green light	Green blinking or steady green light	No display

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Item	Fault	Possible cause	Procedure
1 (a)	LED 1 does not light at all, LED 2 lights yellow and there is a display on LED 3...8	Fuse faulty	Check fuses and exchange if necessary
		Electronic fault	Contact MAQUET Service
1 (b)	LED 1 lights up yellow	Electronic fault	Contact MAQUET Service
1 (c)	LED 2 does not blink green	Batteries already fully charged	Check the charge status by removing mains plug. LED 6 must light
		Electronic fault	Contact MAQUET Service
1 (d)	There is a display in LED 3...8	Electronic fault	Contact MAQUET Service
2 (a)	LED 2 does not light yellow	Battery fault	Change Batteries
		Electronic fault	Contact MAQUET Service
2 (b)	One of the red LEDs 3 or 4 lights up	Battery fault	Order new original batteries
		Electronic fault	Contact MAQUET Service
2 (c)	Light goes out by removing mains plug (Switching over to battery supply)	batteries faulty or incorrectly connected	Check connections and change batteries if necessary.
		G-Fuse on charging electronics faulty	Change fuse (T 2,5 A H)
3		see 1 (a) - (e)	

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2.6 3 monthly check

NOTE:
To guarantee a faultless function in case of power-failure a 3 monthly check must be done.

Table 2-4e: 3 monthly test				
Item	Point checked	Mains operation LED 1	Battery operation LED 2	Charging mode LED 3...6
1	Switch the lights off and charge batteries at least 24 hours	Steady green light	Green blinking	No display
2	Switch lights off	Steady green light	Green blinking (after 24 hours)	No display
3	Unplug mains	Goes out	Shines yellow	Battery operation. Green LED must shine
4	After 2 hours battery operation	out of function	Shines yellow	battery operation red LEDs 3 u. 4 must not shine green LED 5 od. 6 must shine
5	Plug in mains again	Steady green light	Green blinking	No display

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Tab. 2-5e: Fault finding in the 3 monthly test			
Item	Fault	Possible cause	Procedure
1 (a)	LED 1 does not light at all	Electronic fault	Contact MAQUET Service
1 (b)	LED 1 lights up red	No mains voltage Fuse missing	Inform own electrician Insert fuse as in section 4.5
1 (c)	LED 1 lights up yellow	Electronic fault	Contact MAQUET Service
1 (d)	LED 2 does not blink green	Batteries already fully charged	Check the charge status by removing mains plug. LED 8 must light
		Electronic fault	Contact MAQUET Service
1 (e)	There is a display in LED 3...6	Electronic fault	Contact MAQUET Service
2		see 1 (a) - (e)	
3 (a)	LED 2 does not light up yellow	Batteries faulty or incorrectly connected Connection fault in electronics	Check connections and change batteries if necessary. Contact MAQUET Service
3 (b)	One of the two red LEDs 3 or 4 still shines on after 0,5 hours charging time	Batteries faulty Electronics faulty	Request new original batteries Contact MAQUET Service
3 (c)	Light goes out by removing mains plug (switching over to batterie supply)	Batteries faulty or incorrectly connected G-Fuse on charging electronics faulty Electronics faulty	Check connections and change batteries if necessary. Change fuse (T 2,5 A H). See installation instructions (Id. No: 56351221) Assemble casing and wiring diagram P. 7 Contact MAQUET Service
4		see 3 (a) - (c)	
5		see 1 (a) - (e)	

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3 Maintenance

DANGER OF BURNS:

The lighthouse heats up during operation.

Allow to cool off before starting any maintenance or repair work.

3.1 Maintenance and repair

The HANAULUX blue 80 Hospital has been designed and built to last for a very long time. However, to guarantee perfect and safe operation over long periods, it is necessary to check the lights regularly. Regional regulations must also be observed when doing this.

Inspection by the operator

The HANAULUX blue 80 Hospital is to be inspected by the operator every six months with attention to the following points:

- defects in paint work
- cracks in plastic parts
- deformation in the supporting system

In the case of any problems or damage, please inform our customer services.

Inspection by the Service

The HANAULUX blue 80 Hospital has to be serviced once a year by MAQUET or an authorised customer services office. This service covers the following points:

- performance check test
- electrical safety test

The service man is trained in how to conduct the tests.

3.2 Cleaning/Disinfection

Note:

To avoid damage to plastic and glass parts, do not use any scouring agent, or any cleaning agent which is alkaline, acidic or contains alcohol (e. g. ethanol, propanal, ...) or aldehyde.

- Only disinfect the apparatus when it is cold
- Because there are a great number of cleaning/desinfecting agents on the market, and we do not know their interactions with plastic used, damage cannot be ruled out if other agents are used.
- For cleaning, use a soft, lint-free cloth
- The user has to pay attention to the demands of the national committee for hygiene and disinfection.

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3.3 Sterilising the handles

ATTENTION:

Please immediately replace all damaged handles showing cracks or deformations because these could fall down into the wound.

3.3.1 Assembly / disassembly

- Push in the spherical catch of the handle and pull handle off downwards.
- Push the handle back on until you hear the catch "click"; check for proper seat.

3.3.2 Sterilisation

COMMENT:

If you observe the following points, the handles survive a minimum of 350 steam sterilisation cycles without being damaged:

- Place the handles standing upright and with the open side facing down.
- Do not exceed a sterilisation temperature of 134 °C.
- Avoid letting the handles come into contact with other objects while they are being sterilised.

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4 Repair

DANGER OF BURNS:

The lighthouse heats up during operation. Allow to cool off before starting any repair work.

DANGER OF BODILY HARM

Disconnect the HANAULUX *blue 80* Hospital from the mains and make it voltage-free before starting any repair work.

4.1 Replacing the light bulb *Fig. 4*

ATTENTION:

Please note to use replacement bulbs supplied by Maquet to reach the safe funktion and the technical lighting data. Do not touch the glass body of the halogen bulb with bare fingers.

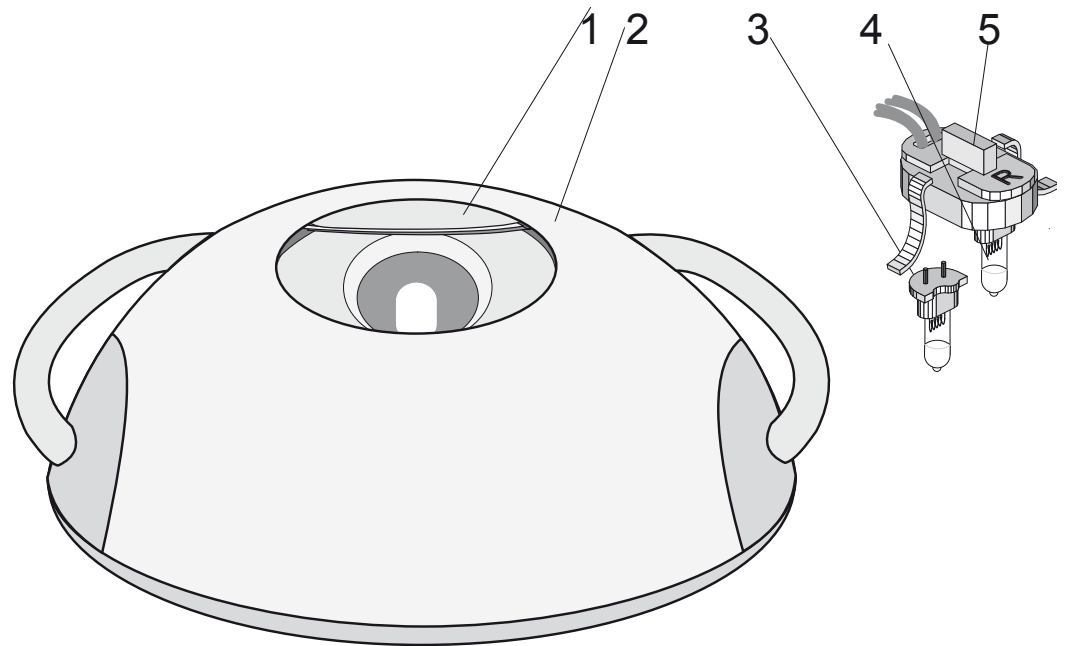
ATTENTION:


When the yellow signal lights up at the lower side of lighthouse *blue 80*, this indicates that the reserve bulb has taken over operation. To avoid a complete failure of the lighting system please replace the main bulb as soon as possible
The position of the main lamp is indicated by a dot on the reflector shield.

1. Slightly push in **lid (1)** on the **hood (2)** and push to one side.
2. Take out **socket (5)** towards to the top.
3. Remove broken **halogen bulb (3)** from the side of the connection lead.
4. Remove **spare lamp (4)** marked with "R" and put into the socket of the main lamp.
5. Put in new bulb into the socket of the spare lamp.
6. Put both lamps back into the reflector in a way that the marked position "R" on the reflector and socket is opposite. Put lid back on and check the lid for safe seat.

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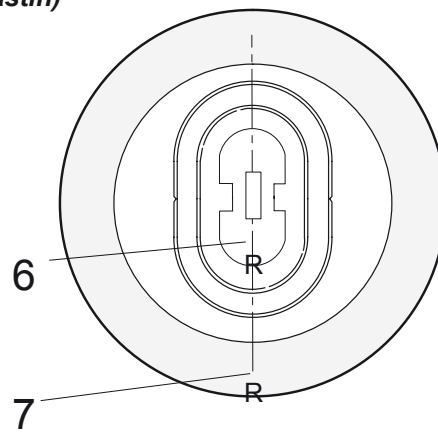4.2 Changing the battery *Fig. 5*

WARNING - ENVIRONMENTAL DAMAGE:

Batteries are to be taken away for raw materials treatment in accordance with local regulations and must not be disposed of in household waste.

4.2.1 Removal

1. Remove **casing cover (1)** by loosening the **10 oval head screws M3 x 8mm (2)**.
2. Dismantle **battery tray (3)** and **connecting lines (4)** and take **batteries (5)** from the casing.

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
COMMENT:

For fitting of batteries, please see the installation instructions for assembly of *blue* 80 Hospital casing, Section 1.2.3 "Fitting batteries"; (Id. No: .56351558).

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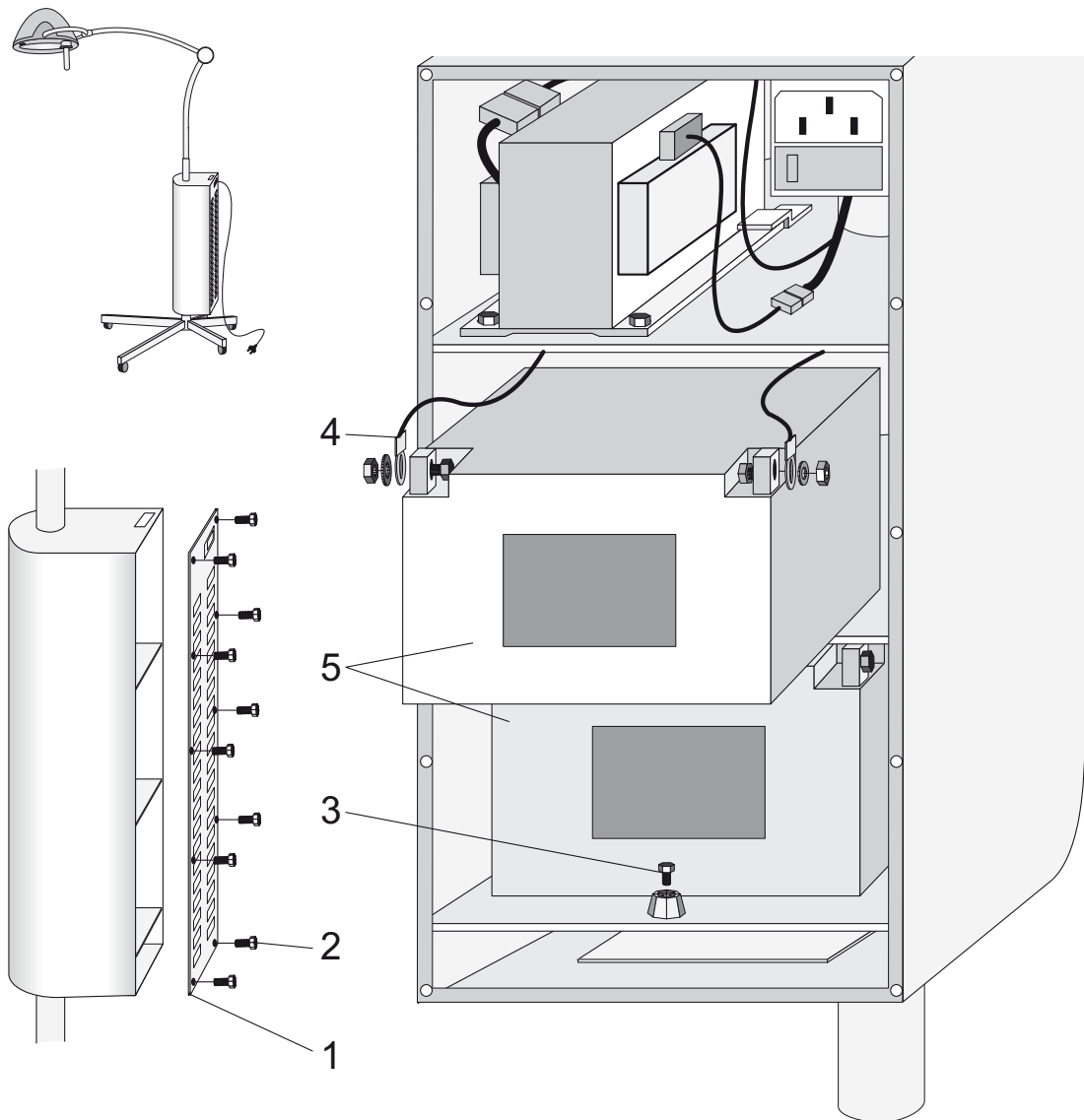Fig. 6

COMMENT:

The brakes of the lighthouse are set during installation. As with mechanical parts the brakes too, are subject to wear and tear.

If the lighthouse no longer automatically maintains every position then it is time to readjust the brakes.

DANGER OF BURNS:

The lighthouse heats up during operation.

Allow to cool off before setting the brake pressure.

1. Switch off lighthouse *blue 80* and allow to cool off.
2. Set the brake pressure of the lighthouse by tightening or loosening **brake pressure adjustment screw (1)** underneath **lighthouse handle (2)**.

4.4 Setting the spring resistance of the counterbalancer

Fig. 7

COMMENT:

Increase or decrease the spring resistance of the counterbalancer if the counterbalancer fails to stop automatically in every position.

1. Remove the **lid of moveable counterbalancer arm (1)** and the **lid of fixed counterbalancer arm (2)**;
2. **Turn adjustment nut (3) by half a turn max.**, make sure to hold the hexagon socket screw on the other side
 - turn clockwise if the lighthouse sinks down
 - turn anti-clockwise if the lighthouse moves up
3. Check whether the light balancer stops moving in every position.
4. Put the lids back on.

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4.5 Adjust stopper on lighthouse Fig. 8

COMMENT:

It is possible to limit the moving range of blue 80 lighthoods in the direction towards the ceiling.

1. Carefully remove the **lid of moveable counterbalancer arm (1)**
2. Move the lighthouse into the desired max. height.

ATTENTION:

Unevenly tightening the threaded bolts may damage the counterbalancer.

Always tighten the threaded bolts evenly.

3. Turn the two **threaded bolts (2)** in until you can feel resistance.
4. Put the lid back on.

4.6 Changing a fuse Fig. 9

Remove the **appliance inlet (1)** below the **socket (2)**.

COMMENT:

The present supply voltage setting can be seen in the unit window (3).

2. Take the grey **insert (4)** out of the **plug-in unit (1)** and insert so that the required supply voltage can be seen in the unit **window (3)**.
3. Insert **two fuses (5)** according to the supply voltage in Table 4-1e.

No.	Supply and voltage	Value
2	115 V and 120 V	5,0 A
2	230 V and 240 V	2,5 A


4. Insert the **plug-in unit (1)** until it is heard to click into position under the socket on the **transformer (2)**.

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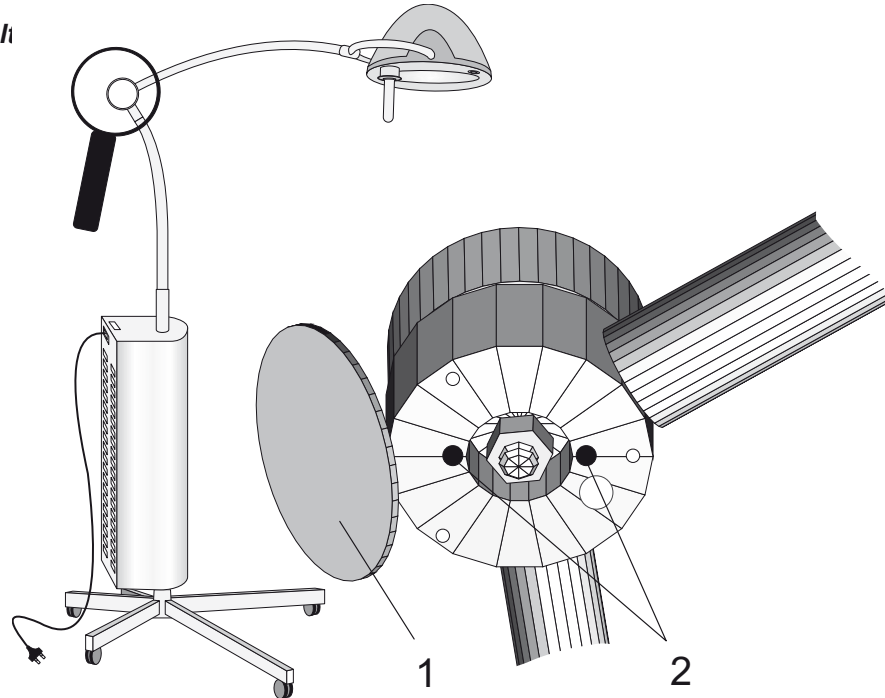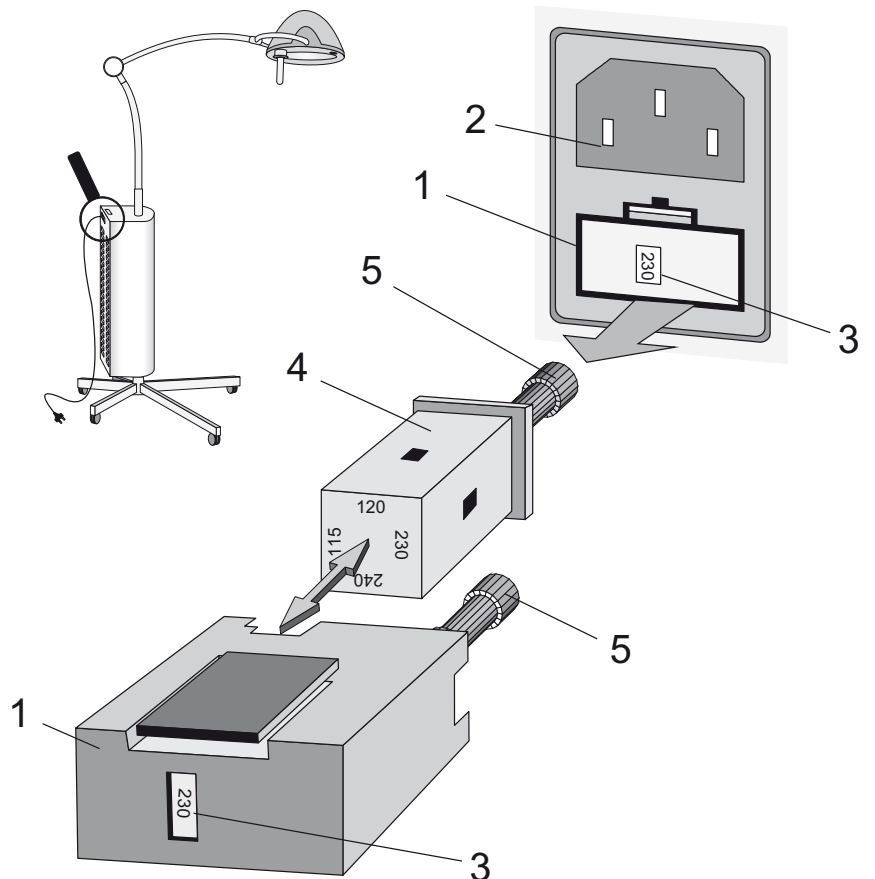
5 Mechanical trouble shooting

Failure	Possible Cause	Corrective measure	Reference
Suspension / Manouverability			
Light is stiff	Brakes on the lighthouse are too tight	Adjust brakes on light head	chap. 4.3
Light is too slack	Brakes on the lighthouse are too loose	Adjust brakes on light head	chap. 4.3
Lighthouse sinks	Spring force in counterbalance too loose	Adjust the spring force in counterbalancer	chap. 4.4
Optical System / Technology			
Light flickers	Faulty contact	Service	chap. 4.1
Illumination too bright / dark	Incorrect supply voltage	Supply voltage to be checked by electrician Check bulb and replace if necessary	chap. 4.1
Life of bulb too short	Wrong bulb(s), Incorrect supply voltage	Check bulb and replace if necessary Supply voltage to be checked by electrician	chap. 4.1
Colour distortion	Wrong bulb(s)	Replace bulb	chap. 4.1
Light field not round	wrong bulb(s), Bulb not positioned correctly,	Check bulb position	chap. 4.1
Glass Cover			
Glass cover scratched or cracked	- Use of wrong cleaning agents; - Cleaning or disinfected when still hot	Use suitable cleaning agent; Glass cover to be replaced by after-sales service	chap. 3.1
Sterilisation			
Life of sterilisable handles too short	Sterilisation process	Check sterilisation process	chap. 3.3
Handles cracked	- Sterilisation process - End of useful life	Check sterilisation process Replace handle	chap. 3.3
If a defect occurs repeatedly or cannot be remedied please contact Maquet.			

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6 Technical data

Technical Lighting Data	HANAULUX <i>blue</i> 80 Hospital
Illumination at distance of 1 meter	80.000 Lux
Light field diameter at distance of 1 meter	18 cm
Working distance	70 - 130 cm
Temperature increase in operating field at nominal illumination	max 13°C*
Temperature increase in head area	2°C
Luminous efficacy	250 lm/W*
Colour temperature	4.300 K
- Automatic switch-over to reserve lamp - All lighting data is based on DIN 5035 Part 3 and Part 6 * Two-stage heat filtering	
Electrical Data	HANAULUX <i>blue</i> 80 Hospital
Supply voltage	AC 115V, 120V, 230V, 240V
Mains frequency	50 / 60Hz
Power consumption	max 150 VA
Bulb	50 W / 24 V
IPS = Internal power supply (Disconnect the unit from the mains by pulling the plug)	
The supply plug serves as the means to disconnect	
Mechanical Data	HANAULUX <i>blue</i> 80 Hospital
Height of field of motion from highest to lowest light head position	ca. 1200 mm
Classification	HANAULUX <i>blue</i> 80 Hospital
Safety class	I
Mode of operation	Continuous operation
IPS = Internal power supply (Disconnect the unit from the mains by pulling the plug)	
Class I in acc. with EU directive 93/42/EWG-MPG	
Approvals <div style="display: flex; justify-content: space-around; align-items: center; margin-top: 10px;">      </div> <p style="text-align: right; margin-top: 5px;">UL 2601-1 CAN/CSA C22.2 NÄ. 601.1 E171155 <8D97></p>	

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